 **BAUERFEIND AG**  
Triebeser Straße 16  
07937 Zeulenroda-Triebes  
Germany  
P +49 (0) 36628 66-20 00  
F +49 (0) 36628 66-29 99  
E info@bauerfeind.com  
**BAUERFEIND.COM**

Rev.12-2021-02\_111 071


## VenoTrain®

Gebrauchsanweisung – medizinische Kompressionsarmstrümpfe  
Instruction for use and care – medical compression arm sleeves  
Conseils d'utilisation et d'entretien – manchons compressifs médicaux  
Las instrucciones de uso y cuidado – medias de compresión de uso médico

 **MADE  
IN  
GERMANY**

All instructions for use:  
[www.bauerfeind.com/downloads](http://www.bauerfeind.com/downloads)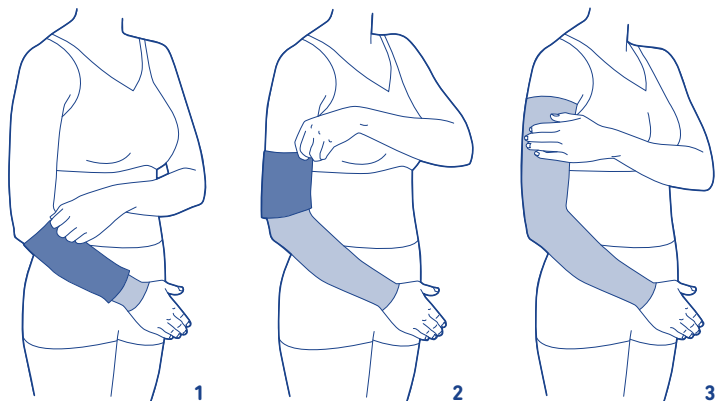

de deutsch.....	3	sr srpski.....	36
en english.....	5	sl slovenski.....	38
fr français.....	7	ro romanian.....	40
nl niederlands.....	9	bg български.....	42
it italiano.....	11	el ελληνικά.....	45
es español.....	14	tr türkçe.....	47
pt português.....	16	ru русский.....	49
sv svenska.....	18	et eesti.....	51
no norsk.....	20	lv latviešu.....	53
fi suomi.....	22	lt lietuvių.....	55
da dansk.....	24	uk українська.....	57
pl polski.....	26	he עברית.....	60
cs česky.....	28	ja 日本語.....	62
sk slovensky.....	30	ko 한국어.....	64
hu magyar.....	32	ar عربي.....	66
hr hrvatski.....	34	zh 中文.....	68

de deutsch

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 english

## Dear Customer,

thank you for choosing a Bauerfeind product. We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

## Intended purpose

VenoTrain are medical products. They are compression stockings for the long-term treatment of lymphological disorders and for the long-term treatment of lipedema.

## Indications

- Lymphedema
- Lipedema from stage II
- Lipo-lymphedema
- Scars

## Part of the body this product is used for

In accordance with the indications

## Risks of using this product

There are no known application risks if you use the product solely according to the indication and provided that you have been measured by a trained specialist and you have been taught the correct method for putting on the compression stockings. In the case of compression stockings with a silicone top band, skin irritation can occur in patients who are hypersensitive to this material. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.

- Avoid contact with ointments, lotions, or substances containing grease or acids.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

## Contraindications

- Advanced peripheral artery disease (if one of these parameters is met: ABPI < 0.5, arterial pressure at the ankle < 60 mmHg, toe pressure < 30 mmHg, or tcpO2 < 20 mmHg at the instep).
- Decompensated heart failure (NYHA III + IV)
- Septic phlebitis
- Phlegmasia coerulea dolens

In the following cases, the decision to treat should be made after weighing the benefits against the risks in order to choose the most appropriate means of compression:

- Dermatitis with significant weeping
- Intolerance of the compression material
- Severely impaired sensitivity of the extremity
- Advanced peripheral neuropathy (e.g. in the case of diabetes mellitus)
- Rheumatoid arthritis

The manufacturer will not accept liability – particularly product liability – in the event of failure to comply with the above contraindications.

## Application instructions

- The initial fitting must only be performed by medically trained specialists.
- It is best to put on your stockings first thing in the morning, as soon as you get up, or after lymph drainage.
- Make sure that you put them on the right way round. The sewn-in label is on the upper end of the sleeve, near the shoulder.
- Wear Bauerfeind gloves to put on and take off your VenoTrain compression stockings as this makes it easier to distribute the material evenly.
- Your arms and hands must be dry.
- Do not grasp the knitted fabric with sharp fingernails. Make sure that you take off your jewelry before starting.
- Please avoid overstretching the stockings and do not pull the product up by the silicone top band.
- Avoid damaging the knitted fabric and do not cut off any loose threads or labels, as this could cause stitches made during production to unravel.
- Store your VenoTrain compression stockings in a dry place away from direct light.
- Compression stockings are articles for daily use, and as such they are subject to natural wear when used on a regular basis. The recommended length of use is six months. This is based on correct handling (putting on and removal, care, etc.). The product's maximum useful life under proper storage conditions is indicated next to an "hourglass" symbol printed on the label on the packet.

## Cleaning instructions

- Please use mild detergent and a lingerie bag. Never expose your product to direct heat. Regular care guarantees optimum effectiveness of the product.
- The product can be machine dried provided a delicate cycle is chosen. **Also observe the**

## Instructions on the sewn-in label on the upper edge of your product.

- When handwashing the stockings, we recommend drying them between two towels (do not wring).
- Avoid the use of washing additives like fabric softener, optical brighteners or stain removers.
- Please wipe the silicone non-slip band on the inside once a day with a damp cloth.

## Notes on reuse

This product is intended for your use only.

## Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the VenoTrain have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

## Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

## Disposal

Once you have finished using the product, please dispose of it according to local specifications.

## Material content

The following materials are used in our compression stockings: polyamide and

elastane. For information regarding the composition in percent, please refer to the individual sewn-in label.

 – Medical device

 – DataMatrix Unique Device Identifier UDI

Version: 2021-02

For more details, please consult your physician or medical retailer.

## Tips for putting on the compression stockings

- Fold the arm sleeve over the cuff at the wrist.
- Pull the folded arm sleeve over the hand.
- Now carefully hold the outside of the knitted fabric and smooth it gradually upward until it sits evenly on the arm.

## Please refer to the instructions provided on the first inside page of this brochure.

The use of the VenoTrain glider donning aid makes it easier to put on your VenoTrain compression stockings.

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Bauerfeind AG
Bauerfeind Middle East FZ LLC
Bauerfeind USA, Inc.