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english

Dear Customer,

thank you for choosing a Bauerfeind product.
We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read our **instructions** for use carefully! If you have any questions, contact your doctor or medical retailer.

Intended purpose

GloboTec comfort sports / GloboTec comfort business are medical devices for the feet. They are orthopedic orthoses that are suitable for use in treating and preventing foot complaints.

Indications

These foot orthoses provide effective support for the treatment of foot deformities and complaints: arch depression / splay foot, pes valgus, pes cavus / high arch foot, complaints of the Achilles tendon, knee or back associated with the foot deformities listed above.

Risks of using this product

The foot orthosis is not suitable for people who have diabetes (diabetic foot syndrome) or rheumatism (rheumatoid foot deformities), as it does not provide soft enough cushioning for these types of foot complaints.

Before using / applying a foot orthosis, you should consult a physician and an orthotist about the risks mentioned above.
If there is not enough space inside the shoe for the foot and the selected foot orthosis, or if the foot orthoses are worn in shoes of an incorrect length and width for shoes with inadequate lateral guidance in the case of open shoe types), problems relating to poor fit may arise (e.g. a feeling of constriction or pressure, rubbing).

If there are pressure points on the edges of the foot orthoses, blisters and calluses may form on the soles of the feet. The same symptoms may occur if excessive mechanical strain leads to holes or creases developing in the top covers, cushioning material or the heel covers, or to the backing of these. You may **stop** wearing foot orthoses if they are damaged in this way!
Creases in stockings or pantyhose must be pulled flat before the shoes are put on, as creases of this kind can cause blisters and pressure points.

Contraindications

Diabetic foot syndrome and rheumatoid foot deformities.

Application instructions for the wearer of the foot orthoses

The intended purpose of the foot orthoses must guarantee sufficient internal volume for both the foot and the selected foot orthosis (adequate shoe length / width, removable, sufficiently thick cushioned sole, adequate lateral guidance of the foot orthoses in the case of open shoe types).

An uncomfortable fit that constricts the foot must always be avoided.
When selecting the correct size of foot orthosis, the packaging information serves merely as a guide. Just like correctly fitting shoes, foot orthoses must be at least 1.0 cm longer than the foot. When the foot is being used, the use of the foot orthoses must be checked during their use. Please check both feet and, if in doubt, base your size selection on the longer foot.

Foot orthoses must always be worn as a pair! Any exceptions to this must be stipulated by your orthotist.
The foot orthosis should be positioned straight in the shoe and as much surface contact as possible with the shoe insole without tilting. Contact between the foot orthosis and the shoe insole helps reduce the risk of pressure marks or blistering on the skin of the soles of the feet.

If you are uncertain about anything during the fitting or regarding the fit, please contact your foot orthotist.
Please regularly check your feet for any pressure marks, redness or other changes caused by the foot orthosis. If you have any problems, please contact your orthotist or your physician.

Cleaning instructions

Foot orthoses must be removed from the shoes on a daily basis in order to avoid moisture to evaporate. The foot orthoses are not machine-washable.
For cleaning materials facing the foot can be wiped if required, using a mild cleanser and a sponge or cloth. After cleaning, any residual soap must be thoroughly rinsed off with water. The orthoses must be left to air dry and no external heat source should be used (heating, hair-dryer, direct sunlight).
Mild disinfectants suitable for human skin may be applied sparingly (sprayed on). However, this may lead to discoloration of top cover materials.

Part of the body this product is used for

The foot orthoses must be worn in conjunction with hosiery in suitable shoes with a sufficient internal volume for both the foot and the selected orthosis.

Notes on reuse

The product is designed to provide care for an individual patient / individual foot orthoses which are not to be reused by any other person. The manufacturer accepts no liability in the event of improper care of the product.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the producer do not apply. Please contact your retailer for more information.
The product in the event of a potential claim under the warranty.
Please do not make any changes to the product yourself! This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.
Discoloration, in particular fading of the top covers of the foot orthoses, may occur during their use. This is dependent on a number of factors, such as moisture levels and sock / stocking materials. This is not covered by warranty.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

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