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## en english

## Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these instructions for use carefully. If you have any questions, please contact your doctor or medical retailer.

### Intended purpose

Trilactive OA is a medical device. It is an orthopedic orthosis for the support and relief of misalignments in the foot as well as osteoarthritis of the knee and hip.

### Indications

• Misalignment of the foot axis and associated functional impairments during heel-to- toe movement

• Medial osteoarthritis of the knee (version with pronation wedge), lateral osteoarthritis of the knee (version with supination wedge)

• Concomitant indications: skew foot / pes valgus or arch decreased / splay foot / mild to moderate pes cavus / high arched foot

The treating physician is responsible for the precise diagnosis.

### ▲ Risks of using this product

The foot orthosis is not suitable for diabetics with PNP and / or PAD or rheumatism (rheumatoid foot deformities), as it does not provide soft enough cushioning for the foot.

Before using / applying a foot orthosis for the risks mentioned above, you should consult a physician or specialist\*. Furthermore, combining the aid with other medical products (supports and compression stockings) may only be considered after having sought advice from your treating physician or specialist.

If there is not enough space inside the shoe for the foot and the selected foot orthosis, or if the product is worn in shoes of an incorrect length and width (or shoes with inadequate lateral guidance in the case of open shoe types), problems relating to poor fit may arise (e.g. a feeling of constriction or pressure).

If the feet are positioned on the edges of the foot orthoses, blisters and calluses may form on the soles of the feet. The same symptoms may occur if excessive mechanical strain leads to blisters or creases developing in the top covers, cushioning material or orthotic cores, or to the breaking of the feet. You must stop wearing foot orthoses if they are damaged in this way!

Creases in stockings or pantyhose must be pulled flat before the shoes are put on, as creases of this kind can cause blisters and pressure points.

### Contraindications

People with diabetes or rheumatoid foot deformities must seek medical advice before using this product.

### Application instructions for the wearer of the foot orthoses

Shoes suitable for wearing with foot orthoses must guarantee sufficient internal volume for both the foot and the selected foot orthosis (adequate shoe length/ width, removable, sufficiently thick cushioned sole, adequate lateral guidance of the foot orthoses in the case of open shoe types). An uncomfortable fit that constricts the foot is to be avoided.

When selecting the correct size of foot orthosis, the packaging information serves merely as a guide. Just like correctly fitting shoes, foot orthoses must be at least selected to fit the foot when the foot is in a weight-bearing, standing position. As feet may differ in length, please check the feet and, if in doubt, base your size selection on the longer foot.

Foot orthoses must always be worn as a pair! Any exceptions to this must be stipulated by your physician.

The foot orthosis should be positioned straight in the shoe and have as much surface contact as possible with the shoe insole. Contact between the foot orthosis and the shoe upper helps reduce the risk of pressure marks or blistering on the skin of the top of the feet.

If you are unsure about anything during the fitting or regarding the fit, please contact a specialist.

The foot orthosis must not be worn continuously for more than 24 hours.

Please regularly check your feet for any pressure marks, redness or other changes caused by the foot orthosis. If you have any problems, please contact a specialist or your physician. Foot orthoses must be checked by a specialist after they have been worn for approximately half a year to ensure that they are still functioning properly.

### Cleaning instructions

Foot orthoses must be removed from the shoes on a daily basis in order to avoid moisture to evaporate. The foot orthoses are not machine-washable, unless expressly indicated otherwise on the packaging.

All surface materials facing the foot can be wiped if required, using a mild cleanser and a sponge or cloth. After cleaning, any residual soap must be thoroughly rinsed off with water. The orthoses must be left to air dry and no external heat source should be used (heating, hair-dryer, direct sunlight).

Mild disinfectants suitable for human skin may be applied sparingly (sprayed on). However, this may lead to discoloration of top cover materials.

### Part of the body this product is used for

The foot orthoses must be worn in conjunction with hosiery in suitable shoes with a sufficient internal volume for both the foot and the selected orthosis.

### Notes on reuse

The product is designed to provide care for an individual patient /individual foot orthoses wearer and must not be reused or used by any other person. The manufacturer accepts no liability in the event of improper use of the product.

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. Please contact the person who gave you the product directly in the case of a problem. You can also claim under the warranty. Please do not make any changes to the product yourself! This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the full therapeutic effectiveness of our products for a long time. Only care that functions optimally can help you. Failure to follow the instructions may also restrict the warranty.

Discoloration, in particular fading of the top covers of the foot orthoses, may occur during use. This is not a defect. Discoloration may be caused, such as moisture levels and sock/stocking materials. This is not covered by warranty.

### Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

### Disposal

Once you have finished using the product, please dispose of it according to local specifications.

## Material content

Polyethylene (PE), Polyester (PES), Thermoplastisches polyurethane (TPU), Polycarbonat (PC), Styrene-butadiene rubber (SBR)

 — Medical Device

 — DataMatrix Unique Device Identifier

Version: 2020-12

\* A specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.

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