 **BAUERFEIND AG**
Triebeser Straße 16
07937 Zeulenroda-Triebes
Germany
P +49 (0) 36628 66-35 00
F +49 (0) 36628 66-39 99
E info@bauerfeind.com
BAUERFEIND.COM

ViscoHeel®

Gebrauchsanweisung
Instruction for use
Notice d'utilisation
Instrucciones de uso

de	deutsch	2
en	english	3
fr	français	5
nl	nederlands	6
it	italiano	7
es	español	9
pt	português	10
sv	svenska	12
no	norsk	13
fi	suomi	14
da	dansk	16
pl	polski	17
cs	česky	18
sk	slovensky	20
hu	magyar	21
hr	hrvatski	22
mk	македонски	24
sr	srpski	25
sl	slovenski	26
ro	romanian	28
el	ελληνικά	29
tr	türkçe	31
ru	русский	32
lv	latviešu	34
lt	lietuvių	35
uk	українська	36
he	עברית	38
ja	日本語	40
ko	한국어	41
ar	عربي	43
zh	中文	45

de deutsch

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en english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

ViscoHeel is a medical device for the feet. It is a viscoelastic heel cushion that provides shock absorption and relief during load peaks. ViscoHeel can be used for redistribution of pressure and pain reduction when treating pathologies in the heel and hindfoot area.

Indications

- Wear to the leg and vertebral joints (shock absorption)
- Knee and hip endoprostheses (shock absorption)
- Irritation of the Achilles tendon and the surrounding tissue (achillodynia – tendomyopathy)
- Haglund's deformity (partial relief)
- Tarsal bone pain

Risks of using this product

⚠ Caution*

ViscoHeel must only be worn in accordance with the specifications in these instructions for use. To ensure the optimum fit of the ViscoHeel, the size required must be determined exactly. Only a trained specialist¹ may carry out the (initial) fitting of the product and provide instruction.

Combination with other products, e.g. for evening out leg lengths, should be discussed beforehand with the physician treating you.

Should you notice any unusual changes (e.g. increased symptoms), please contact your physician without delay. No product liability is accepted in the event of improper use. Do not allow the product to come into contact with ointments, lotions or substances containing grease or acids.

No side effects that affect the entire body have been reported to date. Correct usage / fitting is assumed.

Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:

Skin disorders / injuries to the relevant part of the body, particularly if inflammation, impaired sensation, arterial occlusive vascular disease, or disorders of the lymph system are present. Likewise, any raised scars with swelling, redness, and excessive heat build-up.

Application instructions

Inserting the ViscoHeel

Use the ViscoHeel in shoes with a stiff heel counter. Discuss the use of the ViscoHeel with the physician treating you beforehand, particularly if you want to use it in cushioned running shoes.

Unless otherwise prescribed by your physician, place **both** ViscoHeel heel cushions in your shoes so they fit snugly into the heel area of the shoes. Please take the right and left versions into account when doing this.

Cleaning instructions

Note: Never expose the ViscoHeel to direct heat (e.g. heaters, sunlight, in your car, etc.)! This may cause damage to the material, which can impair the effectiveness of the ViscoHeel.

If necessary, you can clean your ViscoHeel heel cushions with a mild detergent in lukewarm water. Leave the heel cushions to air dry.

Notes on reuse

This product is intended for your treatment only.

Places of use

The foot orthoses must be worn in suitable footwear and with socks or stockings on your feet.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the ViscoHeel have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Silicone (SI)

 – Medical device

 – DataMatrix Unique Device Identifier

Version: 2020-05

* Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the product).

¹ A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of orthoses which are relevant to you.

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Bauerfeind Middle East FZ LLC
Bauerfeind USA, Inc.