AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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(en) english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

ViscoPed is a medical device for the feet. It is a viscoelastic foot orthosis for pressure relief in the forefoot, midfoot and heel area and for gentle correction of static or dynamic misalignment of the skeleton of the foot.

Indications

- Mild skew foot / flat foot deformities
- Splay foot
- Forefoot and toe deformities (around the sole of the foot)
- Pressure pain around the sole of the foot
- Local pressure peaks
- Joint pain

Risks of using this product

⚠ Important Information

- ViscoPed is a prescribed product that should be applied under a physician's guidance. ViscoPed should only be applied in accordance with the instructions contained in these usage directions and for the listed areas of application.
- No liability is accepted for improper use of the product.

- The product should not undergo any improper modification. Failure to comply with the requirement can impair product performance and thus rule out any product liability.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your medical provider without delay.
- Never expose the ViscoPed to direct heat (e.g. heater)! This may cause damage to the material, which can impair the effectiveness of the ViscoPed.
- Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed.

Contra-indications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider: Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up. The inlay is not suitable for diabetics who have diabetic foot syndrome.

Fitting ViscoPed

⚠ ViscoPed must be fitted for the first use by a trained specialist, otherwise we cannot accept any liability for the product.

1. Place the ViscoPed insoles into the shoes as marked (R = right, L = left) so that they are flush with the heel area of the shoes.
2. If the ViscoPed insoles are too large, the front part (near the heel) can be trimmed with a pair of scissors. The insoles are marked accordingly.
3. Place the fitting aid on the insole with the fluffy side downwards and slip foot into the shoe. Remove fitting aid.
4. Unless otherwise prescribed by the physician treating you, insert both the left and right version of the foot orthosis into your shoes.

Product care

We recommend washing ViscoPed handwarm with a mild detergent.

This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer.

Part of the body this product is used for

The foot orthoses must be worn in conjunction with hosiery in suitable shoes with a sufficient internal volume for both the foot and the selected orthosis.

Notes on reuse

The ViscoPed is intended for your treatment only and has been specifically selected according to your measurements. For this reason, do not pass the ViscoPed on to other people.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the

instructions on how to handle and care for the ViscoPed have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

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