AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear customer,

thank you for choosing a Bauerfeind product. We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

AirLoc is a medical device. It is an orthosis for the stabilization of the upper ankle with adjustable air cushioning.

AirLoc must be individually adapted to the patient by a trained orthopaedic technician in order to gain the maximum effect and optimal wear comfort from this orthosis.

Indications

- Treatment of (acute) injuries to the capsular ligament complex
- Early functional / non-surgical treatment, e.g. after ankle sprains and ruptured ligaments
- Post-operative rehabilitation / protection, e.g. after suture repair / ligament reconstruction
- Chronic ligament insufficiency / instability
- Recurrence prevention

Risks of using this product

Important Information²

- AirLoc is a prescribed product that should be applied under medical supervision. AirLoc should be applied in accordance with these directions for use and only for the specified indications.
- No liability is accepted for improper use of the product.
- No improper modifications may be made to the product. Failure to comply with this requirement may adversely affect product

performance, thereby cancelling any product liability.

- The product should be worn in combination with sturdy footwear (e.g. athletic shoes or street shoes).
- Patients with sensitive skin, known contact allergies or who are sensitive to pressure should wear a cotton sock or stocking under the AirLoc.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your medical provider without delay.
- A combination with other products (e.g. compression stockings) must be discussed beforehand with your medical provider.
- Avoid the use of ointments, lotions or any substances including grease or acids.
- Contact sports present a higher risk of injury. Bauerfeind does not guarantee that AirLoc will prevent injuries during these activities.
- Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed. Any supports and orthoses applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.

Note: Never expose the AirLoc to direct heat (e.g. heater, sunlight, in your car, etc.!) This could damage the material. This can impair the effectiveness of the AirLoc.

Contra-indications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Severe varicose veins.
3. Impaired sensation and circulatory disorders of the legs, e.g. Diabetes mellitus.
4. Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.

Application instructions

Fitting and removing AirLoc

Fitting:

- Open all the straps and unfasten the Velcro strap ends.
- Open the valves on the inside and outside of the orthosis.
- Ensure that no air is present in the air cushions.
- Position the orthosis so that the shells labelled "outside" are located on the outer (lateral) side of the leg (Figure 1).
- AirLoc orthoses are labelled "right" and "left" for use on the respective foot.
- Position the shells on the side of the leg so that the front edge of the orthosis is aligned with the edge of the shin (Figure 2).
- Two straps are fitted to each inner and outer shell to secure the orthosis.
- When fitting AirLoc for the first time, start by fastening the upper straps.
- Fasten the straps in the following sequence:
 1. Fasten the anterior upper strap (Figure 3)
 2. Fasten the posterior upper strap (Figure 4)
 3. Fasten the anterior lower strap (Figure 5)
 4. Fasten the posterior lower strap (Figure 6).
- Secure the Velcro ends in the center of the corresponding Velcro section of the

respective strap on the inner / outer edge of the shell.

- When the orthosis is positioned correctly, retighten the two upper straps.
- Now put on your shoe.
- Pump up the air cushions uniformly using the enclosed inflating bulb (Figure 7).
- Start with the outer air cushion followed by the inner cushion.
- The correct degree of stabilization is usually achieved after 1–2 pumps.
- The pressure can be regulated individually via the valves.
- To deflate the cushions, insert the pin on the side of the inflating bulb into the valve to allow the air to escape (Figure 8).
- Only open the two anterior straps when removing the orthosis.
- The two posterior straps do not need to be opened.

Removing:

- Deflate the air cushions.
- Undo the straps, starting with the upper strap and then the lower strap, and remove the orthosis.
- Before refitting the orthosis, ensure that the air cushions are completely deflated.
- The product is intended to be fitted only once and to one patient.
- Guarantee subject to statutory regulations.
- The product can be disposed of without any restrictions.

Cleaning instructions

AirLoc can be washed by hand using a mild detergent. Please close all Velcro® fastenings before washing to prevent subsequent weakening of the Velcro® fastening force.

This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer or our customer service department.

Permissible conditions and places of use

In accordance with the indications (ankle joint).

Assembly and fitting

AirLoc is supplied ready for use. It is available in versions for the right and left foot.

Technical parameters

AirLoc is an ankle orthosis with inflatable air bladders and four adjustable Velcro straps. A bellows is included with the product and available as a spare part.

Notes on reuse

The product is intended to be fitted only once and to one patient.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the AirLoc have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

The product can be disposed of without any restrictions.

Material content

Polypropylene (PP), Polyurethane (PUR), Polyamide (PA), Styrene-ethylene / butylene-styrene block copolymer (TPS-SEBS), White bronze (CuSnZn), Polyester (PES)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Version: 2020-12

¹Orthèse = orthopedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

²Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the product).

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