**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0) 800 4430-130  
F +43 (0) 800 4430-131  
E info@bauerfeind.at

**BENELUX**

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 english

## Dear Customer,

thank you for choosing a Bauerfeind product. We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully.

If you have any questions, please contact your doctor or medical retailer.

## Intended purpose

The CerviLoc and CerviLoc S cervical supports are medical devices. They are orthoses for stabilization and relief of the cervical spine.

## CerviLoc indications

- Pain in the cervical spine area (cervical syndrome)
- Whiplash

## CerviLoc S indications

- Severe pain in the cervical spine area (acute cervical syndrome)
- Moderate whiplash
- Blockage

## Risks of using this product

### **Caution\***

CerviLoc / CerviLoc S must only be worn in accordance with the specifications contained in these instructions and used for the areas of application (purpose, places of use) listed.

No side effects that affect the entire body have been reported to date. Correct fitting is assumed. Any medical aids – supports and orthoses<sup>1</sup> – applied externally to the body can, if tightened excessively, lead to local pressure symptoms or, rarely, constrict the underlying blood vessels or nerves. If this happens, check the fit of your CerviLoc / CerviLoc S, loosen the strap and have the size checked if necessary.

 Contains natural rubber latex (Elastodien) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

**Please note:** never expose the CerviLoc / CerviLoc S to direct heat (e.g. heaters, sunlight, in your car, etc.!) This may cause damage to the material, which can impair the effectiveness of the CerviLoc / CerviLoc S.

## Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and applied after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up
- An artificial opening into the windpipe (tracheostoma)

## Application instructions

A – cervical collar (neck brace)  
B – anatomically contoured chin notch  
C – Velcro strap

### Putting on the orthosis

- 1 Undo the Velcro strap on the cervical collar.
- 2 Hold the cervical collar so that the Velcro strap is on the outside and on the left. Position the chin notch of the cervical collar directly under your chin. Place the right section of the cervical collar at the nape of your neck.
- 3 Fasten the cervical collar by guiding the Velcro strap around the nape of your neck and attaching it to the corresponding piece

of Velcro. The cervical collar must sit firmly against your neck.

## Taking off the orthosis

Undo the Velcro strap on the cervical collar. Take hold of the cervical collar at the chin and remove it by pulling it forward.

## Cleaning instructions

To avoid damaging the CerviLoc / CerviLoc S, close the Velcro fastening before washing it. Wash the cervical collar by hand with a mild detergent. Do not wring it out. Wrap the cervical collar in a towel and carefully press out the water.

## Part of the body this product is used for

In accordance with the indications (cervical spine). See intended use.

## Maintenance information

If handled and cared for correctly, the product is practically maintenance-free.

## Assembly and fitting instructions

The CerviLoc / CerviLoc S is supplied ready for use.

## Technical specifications / parameters, accessories

The orthoses are made up of a foam section with an anatomical chin notch and a Velcro fastening. The CerviLoc S also includes a plastic component integrated into the foam to reinforce it, providing increased stability for the cervical spine.

## Notes on reuse

This product is intended for your treatment only.

## Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the CerviLoc / CerviLoc S have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist<sup>2</sup> were not observed
- The product was modified arbitrarily

## Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

## Disposal

Once you have finished using the product, please dispose of it according to local specifications.

## Material content

Polyurethane (PUR), Cotton (CO), Polyamide (PA), Elastodien (ED), Polyester (PES)

 – Medical device

 – DataMatrix Unique Device Identifier

Version: 2020-08

\* Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the product).

<sup>1</sup> Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

<sup>2</sup> A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of orthoses which are relevant to you.

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