**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0) 800 4430-130  
F +43 (0) 800 4430-131  
E info@bauerfeind.at

**BENELUX**

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 english

## Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

## Purpose

LordoLoc is a medical device. It is an orthosis<sup>1</sup> for the stabilization and relief of the lumbar spine using tensioning straps.

## Indications

- General pain in the lumbar spine (acute and chronic)
- Moderate degeneration / muscular insufficiency of the spine

## Risks of using this product

### Important information

- LordoLoc is a prescribed product that should be worn under a physician’s guidance. LordoLoc should only be worn in accordance with the instructions provided and for the specified areas of application.
- The product should be worn over a T-shirt or an undershirt.
- **LordoLoc must be individually adapted to the patient by a trained specialist<sup>2</sup>. This is the only way to guarantee the full effect and optimal wearing comfort of the orthosis.**
- Incorrect use of the support will terminate any product liability.

- No modifications may be made to the product. Failure to comply with this requirement may adversely affect product performance, thereby terminating any product liability.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your medical provider without delay.
- Please consult your medical provider before using LordoLoc in combination with other products.
- Do not allow the product to come into contact with ointments, lotions or substances containing grease or acids.
- If you have any complaints, please contact your medical retailer only. We would point out that only clean goods can be dealt with.
- Side effects, that affect the entire organism, have not been reported to date. Correct fitting is assumed. Any supports and orthoses applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.

## Contra-indications

Hypersensitive reactions harmful to health have not been reported to date. If you are experiencing any of the following conditions, supports and orthoses such as the LordoLoc should only be fitted and worn after consultation with your medical provider:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Impaired sensation.
- Severe restriction of the performance of heart and lungs (risk of elevated blood pressure if the aid is worn during severe physical exertion).

## Application instructions

### Information for specialists<sup>2</sup>

If necessary, the stays can be adjusted.

### Fitting LordoLoc

- Place the orthosis around the trunk so that the stays are positioned in the center of the back. The tension straps should be at the top **1**.
- Place the fingers in the specially provided loops on the right and left fastener halves and pull the support forward uniformly **2**.
- Press the left half of the fastener against the abdomen and pull the right half over it to secure the Velcro fastening **3**.
- When securing the Velcro fastening, remove the fingers from the left loop first, followed by the right loop **4**.
- Pull the two ends of the upper strap forward to produce the individually desired level of force to reinforce the compression **5**. Secure the ends of the upper strap against the base Velcro fastening **6**. Ensure that the whole Velcro section is fastened over the product so as to avoid contact between the Velcro tapes and clothing or skin.

### Instructions for Product Care

Please use mild detergent and a mesh bag. We recommend closing the Velcro fastenings before washing to preserve their long-term adhesive strength and to prevent damage. Never expose your product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees optimum effectiveness of the product.

### Part of the body this product is used for

In accordance with the indications (lumbar spine). See Intended use.

### Maintenance information

If handled and cared for correctly, the product is practically maintenance-free.

### Assembly and fitting instructions

LordoLoc is supplied ready for use.

## Technical specifications / parameters

The LordoLoc is made from knitted fabric with additional tensioning straps. The back area of the orthosis is reinforced with corset stays. A fastening extension can be inserted to increase the circumference.

## Information about re-use

This product is intended for your personal use only.

## Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists<sup>2</sup>. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

## Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

## Disposal

Once you have finished using the product, please dispose of it according to local specifications.

## Material content

High Grade Steel, Polyamide (PA), Polyurethane (PUR), Cotton (CO)  
Polyethylene (PE), Spandex (EL), Viscose (CV)  
Polyester (PES), Thermoplastic polyurethane (TPU)

 – Medical device

 – DataMatrix Unique Device Identifier

Version: 2020-09

<sup>1</sup>Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

<sup>2</sup>A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of orthoses which are relevant to you.

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