AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

MalleoLoc is a medical device. It is an orthosis for increased stabilization of the upper and lower ankle.

Indications

- Early functional / non-surgical treatment for injuries to the capsular ligament com-

- plex and the bifurcate ligament, e.g. after sprains and ruptured ligaments
- Post-operative protection after ligamental suturing / reconstruction
- Chronic ligament insufficiency
- Relapse prevention

⚠ Application risks

- The use and duration of use must be discussed with the physician who prescribed the MalleoLoc. He will also inform you whether the MalleoLoc can be used simultaneously with other products, and if so, which ones. Please note all the information you received with the MalleoLoc as well as the instructions for use.
- The product may be damaged if it is not used correctly or is altered arbitrarily and, the warranty will not be valid.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, rarely, constrict the underlying blood vessels or nerves.
- If you experience unusual changes connected to the use of the MalleoLoc – e.g. an increase in discomfort – stop using it and contact a physician immediately.

Contraindications

No clinically significant adverse reactions have been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up
- Impaired sensation and circulatory disorder of the legs or feet, e.g. in diabetes mellitus
- Impaired lymph drainage, including soft tissue swelling of uncertain origin located some distance away from the orthosis.

Application instructions

Putting on / taking off the MalleoLoc

Sit down to put on the MalleoLoc.

- 1 Open the two velcro straps. 2 Push the loop of the lower strap down and out of the holder, to loosen the strap. 3 Turn the MalleoLoc so that the straps point inwards. The projection on the lower end of the shell points forwards. 4 Now place the foot centrally in the shell of the MalleoLoc. The inner part of the stay will be located behind the centre of the ankle 4a and the outer part will be in front of the centre of the ankle 4b.
- 5 Close the upper strap by placing it around the ankle joint from the back. When applying the strap, make sure that it sits without folds.

The lower strap is now placed around the foot and lower leg in the shape of a figure of eight. To do this, proceed as follows:

- 6 Place the strap diagonally over the back of the foot, towards the back, around the lower leg. 7 Now guide the strap diagonally forwards once more, to the outer side of the foot, and 8 push the loop from the bottom to the top, into the holder. 9 Fasten the end of the strap upwards onto the part of the strap lying underneath.

Cleaning instructions

Note: Never expose the MalleoLoc to direct heat (e.g. heaters, sunlight, in your car etc.)! This may damage the material. This can impair the effectiveness of the MalleoLoc. For cleaning, remove the straps from the MalleoLoc.

To avoid damage to the product, wipe the shell of the MalleoLoc with a mild detergent only.

Fasten the Velcro fastening. Wash the straps by hand.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Permissible conditions and places of use

In accordance with the indications (ankle joint).

Assembly and fitting instructions

The MalleoLoc is supplied ready for use.

Assembling the MalleoLoc after cleaning:

Fitting the loop:

1 Hold the loop with the tip pointing downwards and the Bauerfeind logo "B" forwards between thumb and forefinger. Now guide the strap with the light gray broad side through the loop from the front.

Fitting the straps:

Place the long strap that you have just fitted with the loop in front of you, with the dark gray side upwards. 1 Guide the strap with the narrow side from the outside inwards, through the lower (closed) part of the MalleoLoc shell.

2 Now guide the short strap with the dark gray side pointing upwards from the inside outwards, through the open part of the MalleoLoc shell.

Technical specifications / parameters

The MalleoLoc is an orthotic support for the ankle with two straps attached to it. The straps are removable. In addition, a loop for fastening is fitted to the lower (longer) strap. The two straps and the loop are available as spare parts.

Notes on reuse

This product is intended for your treatment only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the MalleoLoc have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Ethylene Vinyl Acetate (EVA), Polyoxymethylene (POM), Polypropylene (PP), Polyester (PES), Polyurethane (PUR), Thermoplastic elastomer (TPE)

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