AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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Barrierefreie Version

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AlMg3, Polyamid (PA), Polyurethan (PUR), Polyether, Polyoxymethylen (POM), Polyester (PES)

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 english

Dear customer,

thank you for choosing a Bauerfeind product. We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these instructions for use carefully. If you have any questions, please contact your doctor or medical retailer.

Intended use

The ManuLoc Rhizo long Plus is a medical product. It is an orthosis for immobilizing the entire hand and forearm with a removable finger support and thumb brace.

Indications

- Post-traumatic care (multiple, complex injuries in the hand / finger area)
- Pre-operative and post-operative care, e.g. Dupuytren's contracture, metacarpal fractures
- Complex ligament injuries in the finger / hand area due to trauma
- Paralysis
- Tenosynovitis (severe)
- Sprains / contusions

Risks of using this product

- Once you have been prescribed the ManuLoc Rhizo long Plus, only use it according to your indication and any additional instructions given by a medical specialist¹. Please consult

a specialist or your doctor first if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be avoided.

- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

Contraindications

No clinically significant adverse reactions have been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:

- Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present; likewise, any raised scars with swelling, redness, and excessive heat build-up
- Impaired sensation and circulatory disorders of the arm / hand, e.g. in the case of diabetes mellitus.
- Severe restriction of the performance of the heart and lungs (risk of a rise in blood pressure if the aid is applied during intense physical exertion).
- Impaired lymph drainage – including soft tissue swellings of uncertain origin located away from the fitted aid.

Application instructions

Putting on the ManuLoc Rhizo long Plus

1 Start by loosening the four Velcro straps on the lower arm splint, the Velcro strap on the finger support and the thumb section. **2** Place the ManuLoc Rhizo long Plus on the correct hand. The straps should be located on the top of the lower arm and on the back of the hand. **3** First, close the wide strap in the wrist area, then move up the arm, fastening the straps in order. **4** Fasten the slim strap on the back of the hand, then the strap adjacent to it by wrapping it around the mid-joints of the fingers and velcroing it to the underside of the finger support. **5** You can also fasten this strap in place to position the thumb securely.

Taking off the ManuLoc Rhizo long Plus

Undo the straps on the lower arm splint, starting at the lower arm and finishing at the finger support. Take off the ManuLoc Rhizo long Plus.

Cleaning instructions

Never expose the ManuLoc Rhizo long Plus to direct heat! This can impair the effectiveness. Hand wash the product at 30 °C. Please note the following:

- Remove the Velcro straps if possible, or fasten them securely in position. The stays have ventilation slots and can therefore remain in the product.
 - Air dry the product. Do not dry it in a tumble-dryer.
- Please note that only clean goods can be dealt with.

Part of the body this product is used for

see Indications and Intended purpose

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Technical specifications / parameters

Your product comprises

- a metacarpal forearm splint (I)
- removable finger support (II)
- removable thumb brace (III)
- removable straps on the forearm splint (IV) The thumb brace, the straps, and the finger support are available as spare parts.

Notes on reuse

This product is intended for your treatment only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the ManuLoc Rhizo long Plus have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

AlMg3, Polyamid (PA), Polyurethane (PUR), Polyether, Polyoxymethylene (POM), Polyester (PES)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Version: 2020-09

Specialists

Assembly and fitting instructions

- Only a specialist may conduct the fitting or assembly.
- Check the optimal and individual fit of the ManuLoc Rhizo long Plus at the first fitting, and adjust if necessary.
- Anatomically contoured aluminum stays can be individually adjusted to the patient. To do this, carefully remove the stays from the insertion pockets, shape them to the anatomy of the patient, and reinsert the three stays into the pockets provided.
- To adjust the thumb attachment to the length and position of the patient's thumb, loosen the Velcro fastening between the hand orthosis and thumb attachment (lower Velcro

section). Adjust to the patient's thumb length by moving the attachment (within the channels provided). The thumb abduction angle is set directly by shaping the integrated aluminum stays and by the Velcro fastening on the thumb. Press the lower Velcro fastening down firmly again after making adjustments.

- The finger support is supplied in a neutral position and can be bent into the required position (e.g. the intrinsic plus position) depending on the treatment (at the height of the metacarpophalangeal joints).
- In the case of an assembled orthosis, adapt the dorsal extension angle in the wrist and the palmar flexion angle in the metacarpophalangeal joints. To do this, form the required support with your fingers in order to define the bending edge.
- Practice the correct procedure for putting on the orthosis with the patient.
- No improper modifications may be made to the product. Failure to comply with this requirement may adversely affect the medical effectiveness of the product, thereby voiding any product liability.

¹ A specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses

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