AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
P +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

AchilloTrain Pro is a medical product. It is a support for the relief of Achilles tendon complaints with a friction pad.

 **AchilloTrain Pro must be fitted for the first use by a trained specialist*, otherwise we cannot accept any liability for the product.**

Indications

- Achillodynia (tendinosis, paratendinitis, bursitis subachillea)
- Haglund's deformity
- Chronic, post-traumatic or post-operative irritation (e.g. in the case of Achilles tendon ruptures)
- Inflammatory processes in the Achilles tendon

Risks of using this product

Important information

The benefits of this product are felt particularly during physical activity.

- Please remove the support during longer periods of rest.
- Once you have been prescribed the AchilloTrain Pro, only use it according to your indication and any additional instructions given by a medical specialist. Please consult a specialist or your doctor first if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

 Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

Contra-indications

Hypersensitive reactions harmful to health have not been reported to date.

In the following conditions such aids should only be fitted and worn after consultation with your doctor:

1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Impaired sensation and circulatory disorders of the foot, e.g. Diabetes mellitus.
3. Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.

Fitting AchilloTrain Pro

1. Take hold of the upper edge of the support and pull it over the foot until the heel reaches the blue knitted section .
2. Ensure that the Achilles tendon is securely held by the shaped insert.
3. Adjust the fit of the support as necessary.

Cleaning instructions

Please use mild detergent and a lingerie bag. Never expose the product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees optimum effectiveness of the product.

Notes on reuse

This product is intended for your treatment only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the AchilloTrain Pro have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Silicone (SI), Polyamide (PA), Elastodiene (ED), Viskose (CV), Cotton (CO), Spandex (EL), Polyurethane (PUR), Polyester (PES)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Version: 2020-07

*A specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.

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