 **BAUERFEIND AG**
Triebeser Straße 16
07937 Zeulenroda-Triebes
Germany
P +49 (0) 36628 66-40 00
F +49 (0) 36628 66-44 99
E info@bauerfeind.com
BAUERFEIND.COM

Rev.7 – 2020-11_111 230

GenuPoint

Patellasehnenbandage
Patella tendon strap
Strap pour tendon rotulien
Vendaje para el tendón patelar

 **MADE
IN
GERMANY**

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended use

GenuPoint is a medical device. It is a support for the relief and stabilization of the patellar tendon.

Indications

- Irritation in the patellar tendon area (patellofemoral pain syndrome, patellar chondropathy, patellar tip syndrome / jumper's knee)

Risks of using this product

Caution*

- GenuPoint is a prescribable product. In case you have received it by prescription from your doctor or physiotherapist, the use and duration of use must be agreed with her / him before utilizing the product. In any case, instruction is required by the specialist who provided you with the product. GenuPoint should only be applied in accordance with the instructions contained in these

usage directions and for the listed areas of application.

- Only a trained specialist¹ may carry out the (initial) fitting of the support and provide instruction.
- This product is intended to be fitted only once and to one patient.
- Product liability cannot be assumed in the event of inappropriate use.
- Do not alter the product in any way. Failure to comply with this requirement may adversely affect product performance, thereby canceling any product liability.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your physician without delay.
- Combining the support with other products (e.g. compression stockings) must be discussed beforehand with your physician.
- The product's unique effect is produced primarily during physical activity. The product should be removed during prolonged rest breaks (e.g. sleeping, continuous sitting).
- Avoid contact with ointments, lotions or any substances containing grease and acids.
- For information regarding the options available for returning the packaging to ensure it is correctly recycled, please observe the instructions from the sales outlet where you purchased this product.

Contraindications

No clinically significant adverse reactions have been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Varicose veins (varicosis).
- Impaired sensation and circulatory disorders of the legs (e.g. diabetes mellitus).
- Impaired lymph drainage – including soft tissue swellings of uncertain origin located away from the fitted aid.

Application instructions

Putting on the GenuPoint

The GenuPoint can be worn on the right or left leg. Before putting it on, please loosen both Velcro closures. Open one side completely by pulling the end out of the tab.

Put the GenuPoint on in the center of the kneecap with the semicircular cut-out edge at the top **1**.

Then thread the Velcro strap back through **2**.

Now pull both strap ends at the same time with equal force to prevent the support from twisting. Make sure that the support is sitting correctly and is level with the patellar tendon **3**. You can adjust the pressure on the patellar tendon to suit you, but it must not be uncomfortable.

Now fasten both Velcro ends to the strap **4**.

Depending on your needs and activity level, you can readjust the individual pressure on the patellar tendon.

Taking off the GenuPoint

To take off the support, first loosen both Velcro closures and then remove one side fully from the guide tab. For storage, both strap ends should be fastened again to avoid damage.

Cleaning instructions

The sewn-in label of the GenuPoint, which contains information on the product name, size, manufacturer, washing instructions and CE mark, is located on the strap of the product.

Fasten both ends of the Velcro strap before washing to avoid damage.

Wash the GenuPoint separately by hand using a mild detergent. Never expose the product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees optimum effectiveness of the product.

Notes on reuse

The GenuPoint is intended for your treatment only. It has been adjusted specifically for you. For this reason, do not pass the GenuPoint on to other people.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the GenuPoint have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Silicone (SI), Polyurethane (PUR), Cotton (CO), Polyoxymethylene (POM), Polyester (PES), Spandex (EL)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Specialists¹

General instructions

- Check the optimal and individual fit of the GenuPoint after the first fitting. Make any corrections if necessary.
- Practice the correct procedure for putting on the support with the patient.
- No improper modifications may be made to the product. Failure to comply with this

requirement may adversely affect product performance, thereby voiding any product liability.

- The GenuPoint is intended for the treatment of one patient only.

Version: 2020-11

¹ A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of orthoses which are relevant to you.

* Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the product).

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Contre-indications (contreindicated)

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Bauerfeind Middle East FZ LLC
Bauerfeind USA, Inc.