AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Purpose

OmoTrain S is a medical device. It is a support for the relief and stabilization of the shoulder joint.

Indications

- Chronic, post-traumatic or post-operative irritation
- Prevention / relapse prevention
- Osteoarthritis (joint wear)

Risks of using this product*

The benefits of this product are felt particularly during physical activity.

- Please remove the support during longer periods of rest.
- Once you have been prescribed the OmoTrain S, only use it according to your indication and any additional instructions given by a medical specialist¹. Please consult a specialist or your doctor first if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

 Contains natural rubber latex (elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions such

aids should only be fitted and applied after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up
- Impaired sensation and movement disorders of the arms (e.g. diabetes mellitus)
- Impaired lymph drainage, including soft tissue swellings of uncertain origin located away from the fitted support
- Vein occlusion in the arm (arm vein thrombosis)

Application instructions

Putting on the OmoTrain S

A – Knitted section of the support , B – Velcro strap, C – Eyelet, D – Massage pad (Delta pad)

- 1 Undo the Velcro strap and refasten the tab just behind the eyelet.
- 2 Place the knitted section between the two attached straps. The opening should be pointing toward you. Turn the two straps outward.
- 3 Then guide the affected arm through the knitted section of the support until the shoulder is well covered. The massage pad (Delta pad) should be resting on the bony protuberance at front and back of the shoulder. If this is not the case, adjust the position of the support or the massage pad.
- 4 Use your hand to take hold of the strap loop from the outside and pull the loop over your head and under your arm.
- 5 Loosen the Velcro tab and pull the strap until the required degree of stabilization is achieved. Fasten the strap.

Taking off the OmoTrain S

Undo the strap. To avoid damaging your clothes or the OmoTrain S, fasten the ends of the strap loosely onto the strap underneath it. Take hold of the bottom edge of the support and pull it downward.

Cleaning instructions

Please use mild detergent and a lingerie bag. Never expose the product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees optimum effectiveness of the product.

Close the Velcro fastenings to avoid any damage. Remove the massage pad before washing. Wash the massage pad by hand if required.

Refasten the massage pad after washing. To do this, lay the OmoTrain S with the strap facing upward on the long side of the knitted fabric. Fasten the massage pad onto the knitted section with the long tapered end pointing toward you.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the OmoTrain S have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Styrene-ethylene / Butylene-styrene block copolymer (TPS-SEBS), Elastodiene (ED), Elastane (EL), Polyurethane (PUR), Viscose (CV), Polyoxymethylen (POM), Cotton (CO), Polyester (PES)

 – Medical device

 – DataMatrix Unique Device Identifier UDI

Status of information: 2021-04

* Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the product).

¹ A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of supports which are relevant to you.

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