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Dear patient,

Thank you for choosing a Bauerfeind product. We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these instructions for use carefully. If you have any questions, please contact your physician or medical retailer.

Intended purpose

The Compression Sock Merino is a medical device. They are compression stockings for the long-term treatment of phlebological and/or lymphatic diseases.

Indications

Chronic venous disorders

- Varicose veins / initial period following varicose vein therapy
- Improvement of venous symptoms
- Prevention and treatment of venous edema
- Venous malformations
- Functional venous insufficiency (obesity, jobs involving sitting or standing)

• Improvement in quality of life for chronic venous disorders

• Prevention and treatment of venous skin changes

• Eczema and pigmentation

• Lipodermatosclerosis and atrophie blanche

• Prevention of recurring venous leg ulcers (CEAP stage 5)

Thromboembolism

- Superficial vein thrombosis
- Deep leg vein thrombosis
- Condition following thrombosis
- Post-thrombotic syndrome
- Thrombosis prophylaxis in mobile patients

Edema

- Lymphedema
- Phlebolymphedemas
- Edema in pregnancy
- Post-traumatic edema
- Post-operative edema
- Post-operative reperfusion edema
- Idiopathic cyclical edema
- Lipedema
- Lipo-lymphedema
- Congestion conditions due to immobility (arthrogenic congestive syndrome, partial or complete paralysis of the extremities)
- Occupational edema (standing or seated occupations)
- Drug-induced edema, if no adjustment is possible

Other indications

- Obesity with functional venous insufficiency
- Inflammatory dermatosis of the leg
- Nausea and dizziness in pregnancy
- Congestion complaints in pregnancy
- Condition following burns
- Scar treatment

Part of the body this product is used for

In accordance with the indications

Risks of using this product

There are no known application risks if you use the product solely according to the indication and provided that you have been measured by a professional and you have been taught the correct method for donning the compression stockings. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.

• Avoid contact with ointments, lotions, or substances containing grease or acids.

• If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your physician.

• Check the specific ground characteristics and ensure the ground is not slippery whenever walking, standing or otherwise stressing the affected part of the body. If the ground is smooth, there is an acute risk of slipping and thus injury!

Contraindications

- Advanced peripheral artery disease (if one of these parameters is met: ABPI <0.5, arterial pressure at the ankle <60 mmHg, toe pressure <30 mmHg, or ttpO2 <20 mmHg at the instep)
- Decompensated heart failure (NYHA III + IV)
- Septic phlebitis
- Phlegmasia coerulea dolens

In the following cases, the treatment decision should be made after weighing the benefits against the risks in order to choose the most appropriate means of compression:

- Pronounced dermatosis with weeping
- Intolerance of the compression material
- Severely impaired sensitivity of the extremity
- Advanced peripheral neuropathy (e.g. in the case of diabetes mellitus)
- Primary chronic polyarthrit

If you have any questions about this information, please contact your physician or professional¹.

The manufacturer will not accept liability – particularly product liability – in the event of failure to comply with the above contraindications.

Application instructions

- Only a professional¹ may carry out the initial fitting of the product.
- It is best to don your Compression Socks first thing in the morning, as soon as you get up, or after lymph drainage.
- Wear Bauerfeind gloves to don and doff your Compression Socks as this makes it easier to distribute the material evenly.
- Your legs must be dry.

• Do not grasp the knitted fabric with sharp fingernails. Make sure that you doff your jewelry before starting.

• Please avoid overstretching the stockings and do not pull the product up by the band.

• Avoid damaging the knitted fabric and do not cut off any loose threads, as this could cause stitches made during production to unravel.

• Store your Compression Sock Merino in a dry place away from direct light.

• Compression stockings are articles for daily use, and as such they are subject to natural wear when used on a regular basis. The recommended length of use is six months. This is based on correct handling (donning and doffing of the product, care, etc.). The product's maximum useful life under proper storage conditions is indicated next to an "hourglass" symbol printed on the packaging label.

• Please use mild detergent and a mesh bag.

• Never expose your product to direct heat. Regular care guarantees the ideal effectiveness of the product.

Cleaning instructions

- Please use mild detergent and a mesh bag.
- Never expose your product to direct heat. Regular care guarantees the ideal effectiveness of the product.

• Also observe the instructions on the sewn-in label.

• When handwashing the stockings, we recommend drying them between two towels (do not wring).

• Avoid the use of washing additives like fabric softener, optical brighteners or stain removers.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. The warranty may be limited or excluded if the instructions on how to handle and care for the Compression Socks have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the professional were not observed
- The product was modified arbitrarily

Disclaimer

For use of the Compression Sock Merino, the following applies: Do not diagnose yourself or decide on which medication to take unless you are a professional¹. Before first using our product, please seek the advice of this professional¹, as this is the only way to assess the effect of our Compression Sock Merino on your body and determine the potential risks of using this product due to your personal constitution.

Follow the advice of the professional¹ as well as all instructions in this brochure or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the professional, please contact your physician or distributor, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

45% wool (merino wool), 44% polyamide, 11% elastane

 – Medical device

 – DataMatrix Unique Device Identifier UDI

Date of information: 2025-06

For more details, please consult your physician or medical retailer.

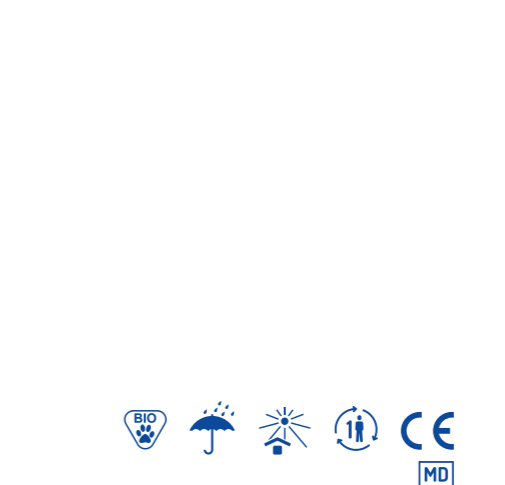
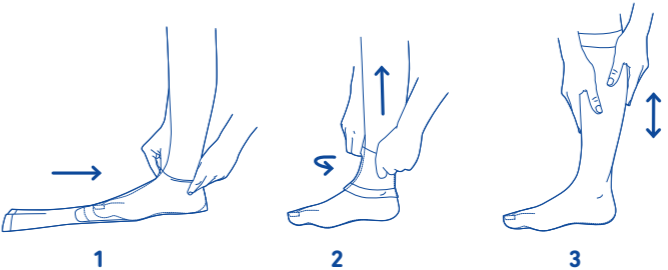
Tips for donning

Closed toe

- 1) Turn the stocking "inside-out" up to the heel. Pull the foot part over your toes and then over the instep.
- 2) Fold the leg part back over and pull it over the heel at the same time. Make sure that the heel is in the correct position.
- 3) Now gradually slide the stocking upward and distribute it evenly until there are no creases. Never pull from the top or the top band.

¹ A professional is any person who is authorized according to the state regulations for fitting and instruction in the use of medical compression stockings which are relevant to them.

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