AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear patient,

Thank you for placing your trust in Bauerfeind. By purchasing the VenoTrain glider plus donning aid, you have selected a product that meets very high medical and quality standards. The excellent workmanship provides you with the convenience you require to don your compression stockings quickly and accurately, day after day.

Please read all of these instructions for use and observe them carefully as they contain important information about how to use your donning aid. Please keep the instructions for use safe, as you may want to read them again later. If you have any questions, please contact your physician or medical retailer.

Intended purpose

The VenoTrain glider plus is an accessory to a medical device. This product makes donning of medical compression stockings for the lower extremities (up to compression class 3) easier.

Indications

- Restricted mobility
- Highly inflammatory or significantly degenerative joint conditions
- Paralysis symptoms
- Conditions of the wrist involving deformity
- Results of injury, amputations
- Malformations
- Extensive stiffening in the spine, knee and hip joints
- Morbid obesity

Risks of using this product

Please observe the specifications in these instructions for use and the instructions given by the professional*.

The VenoTrain glider plus donning aid must only be used in accordance with the instructions in this user guide and the intended use given.

There are no known risks if the product is used properly. Compression stockings with a compression class of more than 3 can cause the donning aid to become deformed. Incorrect storage or direct heat can heat up and change the surface of the donning aid, which can lead to injury.

Contraindications

No contraindications have been reported for the product.

Application instructions

1 Place the VenoTrain donning aid on a firm surface with the opening of the semicircular basket pointing forward. Straighten out the compression stocking and hold it in the basket with the toe pointing forward and the heel pointing toward you. Fold the top band of the compression stocking over the basket with both hands and pull the material over the top half of the basket until the heel appears at the upper edge of the frame. Please make sure that the stocking is pulled straight and is not twisted. **Tip:** Insert your hand into the toe to ensure that it is in the perfect position.

2 Now unfold the handles until they are upright.

3 Position the donning aid on the floor in front of you. To don your stockings safely, we recommend that you use the donning aid sitting down.

Now slide your toes into the foot of the stocking that you have opened up.

Push your foot through until you can place it firmly on the floor. Please check that the toe and heel of the compression stocking are in the correct position.

4 Clasp the handles and slowly pull the donning aid toward your knee.

When you have reached the bottom of your knee, tilt the donning aid backward, remove it from your leg, and put it to one side.

5 Finally, gradually pull the compression stocking upward with both hands and spread out the material evenly. Please make sure

that you don the stocking carefully and do not pull it hard.¹

Gently smooth down the stocking on the leg and avoid creating creases.

Tip: Use our VenoTrain rubber gloves for a light and gentle grip.

Cleaning instructions

Please use a damp cloth or standard disinfectant to clean the donning aid.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Assembly and fitting instructions

The VenoTrain glider plus is supplied ready for use and can be used immediately.

Technical specifications / parameters, accessories

The donning aid is a metal wire frame.

Warranty

The statutory regulations of the country of purchase apply. Where relevant, the country-specific warranty regulations between the distributor and buyer apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty.

Please do not make any alterations to the product yourself. This particularly applies to products that have been individually adjusted by a professional. Please follow our instructions for use and care. They are based on years of experience and ensure that our medically effective products continue to work for a long time. Only aids that function perfectly will help you. Failure to follow the instructions may also limit the warranty.

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a professional. Before first using our medical device, please seek the advice of a physician or professional, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this professional as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics, etc.).

If you still have doubts after your consultation with the professional, please contact your physician or distributor, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Steel, coated, Polyamide (PA)

 – Medical device

 – DataMatrix Unique Device Identifier UDI

Date of information: 2025-08

Our quality promise

 This item was checked thoroughly before being packed. However, should you have any complaints, please return the product to your medical retailer.

*A professional is any person who is authorized according to the state regulations for fitting and instruction in the use of compression stockings which are relevant to them.

¹Please also take note of the instructions for use for your compression stockings.

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