AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear patient,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

VenoTrain ulcertec is a medical device used for the long-term treatment of severe phlebological disorders in the legs.

Indications

AD indications:

- Active venous leg ulcer (CEAP stage 6)
- Prevention of recurring venous leg ulcers (CEAP stage 5)

AG indications:

- Active venous leg ulcer with edema / tendency to edema formation on the thigh
- Venous leg ulcer with varicose veins in the thigh
- Post-thrombotic syndrome on more than one level (pelvis and / or thigh and / or lower leg)

Part of the body this product is used for

On the leg, in accordance with the indications

Risks of using this product

Please closely observe the specifications in these instructions for use and the instructions given by the specialist.

There are no known application risks if the product is used properly, provided your legs have been measured by a specialist* and you have been taught the correct method for donning the compression stockings.

Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.

- Avoid contact with ointments, lotions, or substances containing grease or acids.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

Contraindications

- Advanced peripheral artery disease (if one of these parameters is met: ABPI <0.5, arterial pressure at the ankle <60 mmHg, toe pressure <30 mmHg, or tcpO2 <20 mmHg at the instep).
- Decompensated heart failure (NYHA III + IV)
- Septic phlebitis
- Phlegmasia coerulea dolens

In the following cases, the decision to treat should be made after weighing the benefits against the risks in order to choose the most appropriate means of compression:

- Dermatosis with significant weeping
- Intolerance of the compression material
- Severely impaired sensitivity of the extremity
- Pronounced polyneuropathy (e.g. in cases of diabetes mellitus)
- Primary chronic rheumatoid arthritis in cases of joint pain with compression

If you have any questions about this information, please contact your physician or specialist*.

The manufacturer will not accept liability – particularly product liability – in the event of failure to comply with the above contraindications.

Application instructions

- The initial fitting must only be performed by specialists*.
- It is best to put on your stockings first thing in the morning, as soon as you get up.
- Do not use the compression stockings without a wound dressing.
- Wear Bauerfeind gloves to put on and take off your VenoTrain compression stockings as this makes it easier to distribute the material evenly over the leg.
- Your legs and hands must be dry.
- Do not grasp the knitted fabric with sharp fingernails. Make sure that you take off your jewelry before starting.
- Avoid damaging the knitted fabric and do not cut off any loose threads or labels, as this could cause stitches made during production to unravel.
- Store your VenoTrain compression stockings in a dry place away from direct light.
- The product comes as a set comprising two white liners exerting a low level of compression and a skin-colored overstocking offering a high level of compression. Since the white liner also holds the wound dressing in place, it is advisable to wear it at night too.
- Compression stockings are articles for daily use, and as such they are subject to natural wear when used on a regular basis. The recommended length of use is six months. This is based on correct handling (putting on and removal, care, etc.). The product's maximum useful life under proper storage conditions is indicated next to an "hourglass" symbol printed on the label on the packet.
- The edges of the overstocking and the liner must not be positioned directly on top of each other.

Cleaning instructions

- Please use mild detergent and a lingerie bag. Never expose your product to direct heat. Regular care guarantees optimum effectiveness of the product.
- The product can be machine dried provided a delicate cycle is chosen. **Also observe the instructions on the sewn-in label on the upper edge of your product.**
- When handwashing the stockings, we recommend drying them between two towels (do not wring).
- Avoid the use of washing additives like fabric softener, optical brighteners or stain removers.
- Avoid cleaning with chemicals and ironing.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the VenoTrain ulcertec have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a specialist*. Before first using our medical device, please seek the advice of a physician or specialist* as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical product to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

The VenoTrain ulcertec is made of polyamide and elastane. For information regarding the composition in percent, please refer to the individual sewn-in label. The AG liner additionally has a silicone top band on the upper edge of the stocking. It consists of polyamide, elastane, and silicone.

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Version: 2026-04

* A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of compression stockings which are relevant to you.

Tips for putting on the compression stockings

- A ▶ After applying the wound dressing, gather the white **liner** over your toes.
- B ▶ Stretch the opening for the foot with both hands. Now slip the stocking on until it covers the heel.
- C ▶ When you pull it upward, the stocking should be placed carefully over the wound dressing.
- D1 D2 ▶ Then pull the stocking evenly upward to below the knee.

When you put on the overstocking, we recommend that you use the blue donning aid provided.

- E ▶ To make it easier to put on the **overstocking**, we recommend gathering it up and stretching it horizontally.
- F ▶ Then slip the opening of the stocking over your toes as usual.
- G ▶ Once you have put the stocking on your foot, check whether the heel fits correctly.
- H ▶ You should now pull the stocking upward as far as possible until it reaches the hollow of the knee.
- I1 I2 ▶ This will ensure that it fits correctly and has an optimum medical effect. Make sure that the edging of the overstocking is positioned beneath the band of the liner.

Please refer to the information provided on page 2 of this brochure.

For more details, please consult your physician or medical retailer.

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