AUSTRIA
 Bauerfeind Ges.m.b.H.
 Hainburger Straße 33
 1030 Wien
 P +43 (0) 800 4430-130
 F +43 (0) 800 4430-131
 E info@bauerfeind.at

FRANCE
 Bauerfeind France S.A.R.L.
 B.P. 59258
 95957 Roissy CDG Cedex
 P +33 (0) 1 4863-2896
 F +33 (0) 1 4863-2963
 E info@bauerfeind.fr

SLOVENIA
 Bauerfeind d.o.o.
 Dolenjska cesta 242 b
 1000 Ljubljana
 P +386 (0) 1 4272-941
 F +386 (0) 1 4272-951
 E info@bauerfeind.si

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en english

Dear patient,

please read and observe these instructions for use carefully and if you have any questions, contact your physician or medical retailer or visit info@bauerfeind.com.

Intended purpose

ErgoPad ball & racket is a medical device for the feet. It is a supportive foot orthosis that makes sports shoes more comfortable to wear and helps stabilize the movement of the hindfoot during athletic activities.

Suitable for: Racket and other sports, such as volleyball, handball, basketball, tennis, badminton, squash, golf, football (depending on footwear).

Indications

The ErgoPad ball & racket foot orthosis can be used for the prevention and concomitant treatment of sports-specific conditions associated with excessive strain:

- Non-specific foot problems associated with excessive strain
- Chronic heel and ankle pain
- Achilles tendon problems
- Tibia problems
- Knee problems

It is also used for the prevention and concomitant treatment of sports-related complaints in case of the following foot conditions:

- Mild to pronounced arch decreased / splay foot or
- Mild high-arched foot

Risks of using this product

⚠ Caution: Before using / applying a foot orthosis, you should consult a physician or specialist¹ on the risks involved.

Please closely observe the specifications in these instructions for use and the instructions given by the specialist.

If the available internal volume of the shoe is not sufficient for the foot and the selected foot orthosis, or if the foot orthoses are worn in shoes of an incorrect length and width (or shoes with inadequate lateral guidance in the case of open shoe types), problems relating to poor fit may occur (e.g. a feeling of constriction or pressure, rubbing).

Please remember that shoes and foot orthoses form a single functional unit. If the feet are positioned on the edges of the foot orthosis, blisters and calluses may form on the soles of the feet. The same symptoms may occur if excessive mechanical strain leads to holes or creases developing in the top cover and / or cushioning material, or causes them to break. Foot orthoses that are damaged in this way must no longer be worn!

Contraindications

People with diabetes or rheumatoid foot deformities must seek medical advice before using this product to ensure optimum treatment.

Application instructions

Shoes suitable for wearing with foot orthoses must have sufficient internal volume for both the foot and the selected foot orthosis (adequate shoe length / width and, if necessary, a removable, sufficiently thick cushioned sole, as well as lateral guidance for the foot orthoses in the case of open shoe types). An uncomfortable fit that constricts the foot must always be avoided. **a**

When selecting the correct size of foot orthoses, the packaging information serves merely as a guide. Just like correctly fitting shoes, foot orthoses must be at least 1.0 cm longer than the foot, when the foot is in a weight-bearing, standing position. Please check the length allowance for both feet, as they may differ in length. In case of doubt, base your size selection on the longer foot. **b**

The foot orthosis must be positioned straight in the shoe and have as much surface contact as possible with the shoe insole. Contact between the foot orthosis and the shoe upper helps reduce the risk of pressure marks or blistering on the skin of the soles of the feet. **c**

Foot orthoses must always be worn as a pair! Any exceptions to this must be stipulated by the treating physician. The ErgoPad ball & racket is designed to be worn repeatedly by one user only. Please regularly check your feet for any pressure marks, redness or other changes caused by the foot orthosis. If you have any questions, please contact a specialist or your physician.

Adjustment phase: As with many medical products, the ErgoPad ball & racket may feel somewhat strange to wear at first. We therefore recommend that you start by wearing it for around one hour at a time and slowly increase the wearing period until the foot orthosis feels comfortable. It should not

take much longer than a week to adjust to wearing the foot orthosis. If you still find it uncomfortable to wear after this period, please contact a specialist.

Foot orthoses must be checked by a specialist after they have been worn for approximately half a year to ensure that they are still functioning properly. No side effects that affect the entire body have been reported to date. This assumes the orthosis is used / fitted correctly.

Cleaning instructions

The ErgoPad ball & racket can be cleaned by hand using soap and lukewarm water. Then leave the foot orthoses to air dry. Do not use any external heat sources (e.g. heaters, hairdryers or direct sunlight). The foot orthoses cannot be washed in a washing machine.

Mild disinfectants suitable for human skin may be applied sparingly (sprayed on). However, this may lead to discoloration of the top cover materials.

Part of the body this product is used for

The foot orthoses must be worn in suitable footwear and with socks or stockings on your feet.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Discoloration, in particular fading of the top covers of the foot orthoses, may occur during their use. This is dependent on a number of factors, such as moisture levels and sock / stocking materials. This is not covered by warranty.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

You can dispose of the product in accordance with national legal requirements.

SPECIALISTS

Modification instructions Fitting

For an optimum fit, there should be a length allowance of approximately 1 cm between the foot and the foot orthosis and / or shoe, in a standing position. If the foot orthoses are not the right size for your shoes, you can cut them down or reshape them using thermoforming. Please follow the instructions below.

Cutting

1. Take the inlays out of your sports shoes and place them in your ErgoPad ball & racket.
2. Align the heel sections so that they are positioned precisely on top of one another and draw around the inlay on your ErgoPad ball & racket.
3. Cut along the outline you have drawn in the forefoot area. (Make sure you do not cut the foot orthosis too small. We recommend cutting off less material than you need to at first and trying on the foot orthosis before carrying on cutting).
4. Place the ErgoPad ball & racket in your shoes (foot orthoses for left / right feet are marked as such on the back).

Thermoforming **d**

In exceptional cases and having sought specialist advice, it may be necessary to use thermoforming to adapt the foot orthoses to suit the shape of your feet.

1. After cutting down your ErgoPad ball & racket, preheat your oven to 90 °C / 200 °F.
2. Put the foot orthoses (with the top covers facing downward on a clean surface) in the oven for approximately 2 minutes.

Risk of burning!

Wear protective gloves when taking the foot orthoses out of the oven.

3. As soon as you have taken the foot orthoses out of the oven, place them immediately in your shoes.
4. Put on your shoes straight away (make sure you are wearing socks or stocking).
5. Then stand for 2 minutes with your feet shoulder-width apart and pointing forward.

Your foot orthoses should now be adapted to the shape of your feet.

Material content

Polyester (PES), Polyurethane (PUR)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Version: 2025-01

¹ A specialist is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

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