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en english

Dear patient,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these instructions carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

GlobTec Junior is a medical device for the feet. It is an orthopedic shell foot orthosis suitable for treating foot deformities in children and adolescents until their body growth is completed.

Indications

GlobTec Junior foot orthoses effectively support the treatment of foot deformities. The following indications are addressed:

- Hollow foot (pes cavus), also flattened
- Skew foot (pes valgus)
- Pediatric skinned flat foot (pes planovalgus), which cannot be compensated for
- Congenital flat foot / arch-decreased foot (pes planus)

Pathological leg axis development in children with foot deformity

Risks of using this product

Before using / applying a foot orthosis, you should consult a physician or specialist or the risks mentioned above.

If there is not enough space inside the shoe for the foot and the selected foot orthosis, or if the foot orthoses are worn in shoes of an incorrect length and width (or if the inadequate lateral guidance in the case of open shoe types), problems relating to poor fit may arise (e.g. a feeling of constriction or pressure, rubbing). If the feet are positioned on the edges of the foot orthoses, blisters and calluses may also occur. Please do not use the shoe if the foot orthosis is damaged or if the sole of the shoe upper helps reduce the risk of pressure marks or blistering on the skin of the soles of the feet.

Creases in stockings or pantyhose must be pulled flat before the shoes are put on, as creases of this kind can cause blisters and pressure points.

Contraindications

Weak pediatric skewed flat foot (pes planovalgus), compensated for when standing on toes

Application instructions for the wearer of the foot orthoses

The foot orthosis is exclusively intended for use by growing children following the manufacturer's instructions for understanding indications:

- Hollow foot (pes cavus), also dorgrückt
- Skew foot (pes valgus)
- Kindly skinned flat foot (pes planovalgus), which cannot be compensated for
- Congenital flat foot / arch-decreased foot (pes planus)

When selecting the correct size of foot orthoses, the packaging information serves only as a guide. Please do not use the foot orthosis, if you are unsure which size of foot orthosis is best for your child. If you are unsure, please contact your specialist. The foot orthosis should always lie flat at the midfoot (b1) and hindfoot (b2) – without tilting.

The foot orthosis must always be worn as a pair! Any exceptions to this must be stipulated by your physician.

The foot orthosis should be positioned straight in the shoe and also has much surface contact between the foot orthosis and the shoe. If the shoe upper helps reduce the risk of pressure marks or blistering on the skin of the soles of the feet.

If you are unsure about anything during the fitting or regarding the fit, please contact your local specialist.

Please regularly check your child's feet for any pressure marks, redness or other changes caused by the foot orthosis. If you have any problems, please contact a specialist or your physician.

Adjustment phase

Like many medical products, foot orthoses may feel somewhat strange to wear at first. We therefore recommend that your child starts by wearing the foot orthoses for 1–2 hours at a time and slowly increases the wearing period until it feels comfortable. Normally, it should not take longer than a week to adjust to wearing the foot orthosis. If your child still finds it uncomfortable to wear after this period, please contact a specialist.

The aid supports your child while they are growing and therefore requires particular attention. Please ask your specialist about appropriate check-up intervals.

Cleaning instructions

Foot orthoses must be removed from the shoes on a daily basis in order to avoid moisture to evaporate. The foot orthoses are not machine-washable. All surfaces contacting the foot orthoses must be cleaned with a mild cleanser and a sponge or cloth. After cleaning, any residual soap must be thoroughly rinsed off with water. The orthoses must be left to dry and no external heat should be used when drying. Please use heating, hair-dryer, direct sunlight.

Part of the body this product is used for

The foot orthoses are used in non in conjunction with hosiery in suitable shoes with a sufficient internal volume for both the foot and the selected foot orthosis.

Notes on use

The product is designed to provide care for an individual patient /individual foot orthoses wearer and must not be reused or used by any other person. The manufacturer accepts no liability in the event of improper use of the product.

Warranty

Foot orthoses are designed for daily use and as such they are subject to natural wear when used as a regular basis. Slight discoloring of the particular fading of the top covers of the foot orthoses, may occur during their use. This is dependent on a number of factors, such as moisture levels and sock /stocking materials. This is not covered by warranty.

Please do not return the foot orthoses to the retailer in which you purchased the product directly. Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for

use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty. Please note that foot orthoses wear regularly have a durability of approx. six months.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the orthopedic aids. As the manufacturer, we strongly recommend that you inform us. Our contact details can be found on the back of this brochure.

Disposal

You can dispose of the product in accordance with national legal requirements.

Material content:

Thermoplastisches elastomer (TPE), Polypropylen (PP), Polyester (PES) Polyurethane (PUR), Polyethylen (PE), Ethylen vinyl acetate (EVA)

Version: 2025-07

- Medical Device
- DataMatrix Unique Device Identifier UID

*A specialist is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that you inform us in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

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