**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0) 800 4430-130  
F +43 (0) 800 4430-131  
E info@bauerfeind.at

**BENELUX**

Bauerfeind Benelux B.V.  
waarderveldweg 1  
2031 BK Haarlem  
THE NETHERLANDS  
P +31 (0) 23 531-9427  
F +31 (0) 23 532-1970  
E info@bauerfeind.nl

**BELGIUM**

P +32 (0) 2 527-4060  
F +32 (0) 2 792-5345  
E info@bauerfeind.be

**BOSNIA AND HERZEGOVINA**

Bauerfeind d.o.o.  
Meše Selmovića 19  
71000 Sarajevo  
P +387 (0) 33 710-100  
F +387 (0) 33 619-422  
E info@bauerfeind.ba

**CHINA**

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(en) english

## Dear patient,

Thank you for choosing a Bauerfeind product. We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these instructions for use carefully. If you have any questions, please contact your physician or medical retailer.

### Intended purpose

ViscoPed is a medical device for the feet. It is a viscoelastic foot orthosis for pressure relief in the forefoot, midfoot and heel area and for gentle correction of static or dynamic misalignment of the skeleton of the foot.

### Indications

- Mild planovalgus foot deformities
- Splay foot
- Forefoot and toe deformities (around the sole of the foot)
- Pressure pain around the sole of the foot
- Local pressure pain
- Joint pain

### Risks of using this product

- ViscoPed is a product that should be applied under a physician's guidance. ViscoPed should only be worn in accordance with these instructions for use and for the listed areas of application.
- Product liability cannot be assumed in the event of inappropriate use.
- Should you notice increased symptoms or any unusual changes, please stop using the product and please contact your physician without delay.
- Never expose the ViscoPed to direct heat (e.g. heater)! This may cause damage to the material, which can impair the effectiveness of the ViscoPed.
- Side effects involving the body as a whole have not been reported to date.

## Contraindications

No clinically significant hypersensitivity reactions have been reported to date. If you have any of the following conditions, such aids should only be donned and worn after consultation with your physician:

- Skin disorders / injuries to the relevant part of the body, particularly if inflammation, impaired sensation, arterial occlusive vascular disease, or disorders of the lymph system are present. Likewise, any raised scars with swelling, redness, and excessive heat build-up.
- The inlay is not suitable for diabetics who have diabetic foot syndrome or people with rheumatoid foot deformities.

## Application instructions

 First-time application / fitting of the ViscoPed may only be carried out by professionals<sup>1</sup>, otherwise product liability cannot be assumed.

1. Place the marked ViscoPed foot orthoses (R = right, L = left) in the respective shoes so they fit snugly into the heel area of the shoes.
2. If the ViscoPed foot orthoses are too long, the marked toe area can be shortened using shears.
3. Place the donning aid on the foot orthosis with the fleecy side down and slip into the shoe. Then withdraw the donning aid.
4. Unless otherwise prescribed by the physician treating you, insert both the left and right version of the foot orthosis into your shoes to avoid leg length discrepancy.

## Cleaning instructions

We recommend to wash the ViscoPed in lukewarm water, using a mild detergent.

## Part of the body this product is used for

The foot orthoses must be worn in conjunction with hosiery in suitable shoes with a sufficient internal volume for both the foot and the selected foot orthosis.

## Notes on reuse

This product is intended to be fitted to one patient only.

## Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. The warranty may be limited or excluded if the instructions on how to handle and care for the ViscoPed have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the professional were not observed
- The product was modified arbitrarily

## Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a professional<sup>1</sup>. Before first using our medical device, please seek the advice of a physician or professional<sup>1</sup>, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this professional as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the professional, please contact your physician or distributor, or contact us directly.

## Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

## Disposal

Once you have finished using the product, please dispose of it according to local specifications.

## Material content

Silicone (SI)

 – Medical device

 – DataMatrix Unique Device Identifier UDI

Date of information: 2026-06

1 A professional is any person who is authorized according to the state regulations applicable for fitting and instructing in the use of orthopedic foot orthoses.

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