AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

Bauerfeind Benelux B.V.
Waarderveldweg 1
2031 BK Haarlem
THE NETHERLANDS
P +31 (0) 23 531-9427
F +31 (0) 23 532-1970
E info@bauerfeind.nl
BELGIUM
P +32 (0) 2 527-4060
F +32 (0) 2 792-5345
E info@bauerfeind.be

**BOSNIA AND
HERZEGOVINA**

Bauerfeind d.o.o.
Meše Selmovića 19
71000 Sarajevo
P +387 (0) 33 710-100
F +387 (0) 33 619-422
E info@bauerfeind.ba

CHINA

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 english

Dear patient,

Thank you for choosing a Bauerfeind product. We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your physician or medical retailer.

Intended purpose

The CerviLoc and CerviLoc S cervical supports are medical devices. They are orthoses for stabilization and relief of the cervical spine.

Indications CerviLoc

- Pain in the cervical spine area (cervical syndrome)
- Whiplash

Indications CerviLoc S

- Severe pain in the cervical spine area (severe cervical syndrome)
- Moderate whiplash
- Blockage

Risks of using this product

 **Caution***

Please closely observe the specifications in these instructions for use and the instructions given by the professional.

- CerviLoc / CerviLoc S must only be worn in accordance with the specifications contained in these instructions for use and used for the areas of application listed (intended purpose / parts of the body this product is used for).
- Side effects involving the body as a whole have not been reported to date. Correct usage / donning of the product is assumed. Any aids – supports and orthoses¹ – applied externally to the body can, if tightened

excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves. If this happens, check the fit of your CerviLoc / CerviLoc S, loosen the strap and have the size checked if necessary.

- If you notice any negative changes or an increase in symptoms while using the product, stop any further use and contact your physician.
 - Caution: When loosening the straps or tensioning and fastening systems, or when doffing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.
-  **LATEX** Contains natural rubber latex (elastodiene), which can cause skin irritation in persons with an allergy. Doff the orthosis if you notice any allergic reaction.

Contraindications

No clinically significant hypersensitivity reactions have been reported to date. If you have any of the following conditions, such aids should only be donned and worn after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up
- An artificial opening into the windpipe (tracheostoma)

Application instructions

Please refer to the care instructions on the sewn-in label.

The sewn-in label contains information on the size, manufacturer, care instructions, and CE marking and is located on the inside of the orthosis.

Note: Never expose the CerviLoc / CerviLoc S to direct heat (e.g. heater, sunlight, in your car etc.!) This may cause damage to the material, which can impair the effectiveness of the CerviLoc / CerviLoc S.

Donning

1 Undo the Velcro strap on the cervical orthosis. **2** Hold the cervical orthosis so that the Velcro strap is on the outside and on the left. Position the chin notch of the cervical orthosis directly under your chin. Don the right section of the cervical orthosis at the nape of your neck. **3** Fasten the cervical orthosis by guiding the Velcro strap around the nape of your neck and attaching it to the corresponding piece of Velcro. The cervical orthosis must sit firmly against your neck.

Doffing

Undo the Velcro strap on the cervical orthosis. Take hold of the cervical orthosis at the chin and doff it by pulling it forward.

Cleaning instructions

To avoid damaging the CerviLoc / CerviLoc S, close the Velcro fastening before washing it. Hand-wash the cervical orthosis at 30 °C using mild detergent. Do not wring it out. Wrap the cervical collar in a towel and carefully press out the water.

Part of the body this product is used for

In accordance with the indications (cervical spine). Please see intended purpose.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Assembly and fitting instructions

The CerviLoc / CerviLoc S is supplied ready for use.

Components:

A – Cervical orthosis (neck brace)

B – Anatomically contoured chin notch

C – Velcro strap set

Technical specifications / parameters

The orthoses are made up of a foam section with an anatomical chin notch and a Velcro fastening. The CerviLoc S also includes a plastic component integrated into the foam to reinforce it, providing increased stability for the cervical spine.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. The warranty may be limited or excluded if the instructions on how to handle and care for the CerviLoc / CerviLoc S have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the professional² were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a professional². Before first using our medical device, please seek the advice of a physician or professional² as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this professional as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the professional, please contact your physician or distributor, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

CerviLoc:

Polyurethane (PUR), Cotton (CO), Polyamid (PA), Elastodiene (ED), Polyester (PES)

CerviLoc S:

Polyurethane (PUR), Cotton (CO), Polyester (PES), Polyamid (PA), Elastodiene (ED)

 – Medical device

 – DataMatrix Unique Device Identifier UDI

* Note on the risk of personal injury (risk of injury, harm to health, and accidents) or damage to property (damage to the product).

¹ Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso.

² A professional is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

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