 **BAUERFEIND AG**
Triebeser Straße 16
07937 Zeulenroda-Triebes
Germany
P +49 (0) 36628 66-40 00
F +49 (0) 36628 66-44 99
E info@bauerfeind.com
BAUERFEIND.COM

EpiPoint®

Gebrauchsanweisung
Instructions for use
Notice d'utilisation
Istruzioni per l'uso
Instrucciones de uso

 **MADE
IN
GERMANY**

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 english

Dear patient,

Thank you for choosing a Bauerfeind product.

We wish you a speedy recovery. Please read and observe these instructions for use closely and in their entirety. If there is something you do not understand or if you have any questions, do not use the product until you have consulted your physician or professional¹ again. Keep these instructions for use in case you want to look something up later.

Intended purpose

EpiPoint is a medical device. It is an orthosis² for the targeted relief of the tendon insertions at the elbow.

Indications

- Epicondylitis lateralis humeri (inflammation of the muscle origin, tennis elbow)
- Epicondylitis medialis humeri (inflammation of the muscle origin, golfer's and thrower's elbow)
- (Relapse) prevention

Contraindications

For the following clinical pictures, usage of the product is only indicated upon consultation with your physician:

- Skin disorders/lesions on the part of the body to be treated, especially if inflammatory manifestations as well as raised scars with swelling, redness and overheating are present
- Impaired sensation and circulatory disorders of the forearm and the hand.

- Impaired lymph drainage – including soft tissue swelling of uncertain origin located remotely from the fitted aid.
- Bottleneck syndromes (irritation due to compression of nerves in the arm).

Ask your physician whether your conditions fall into these categories.

Notes before first usage

If you wish to use your Bauerfeind product for acute symptoms, injuries or healed previous conditions, you urgently need to obtain professional medical advice and observe it before using the product. Restrictions may apply to you which you must take into account and which the professional¹ will clarify. In case of identified or suspected problems with blood pressure and intense physical exercise, prolonged use of the product can have additional effects on your blood pressure.

After you have been prescribed/you have purchased the product, only use it according to the indication and under observance of any additional instructions given by your physician/professional¹. If you wish to use the product in combination with medication, other medical devices or other treatments, consult your physician/professional¹ beforehand. If any additional conditions or injuries occur during the period of application of the product, also consult your physician immediately.

Risks of application

- Do not arbitrarily modify the product as this may cause it to not help as expected and/or worsen your health.
- The product must not be applied too firmly as this may cause local pressure symptoms or, in rare cases, constrict underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms related

to the application of the product (such as tissue congestion, skin irritation, swelling, etc.), consult the application and consult your physician/professional¹.

- If you completely doff the product, treatment is insufficient and additional damage to the part of the body affected may occur during additional strain. Be especially careful when this is the case.
- Additionally, restrictions due to the indication may apply which the professional¹ will explain to you. In particular, driving vehicles, other means of transportation or machines may only be possible under restrictions. The same applies to the use of ladders, tools and sports equipment. When in doubt, we recommend to suspend these activities.
- Furthermore, proceed with extreme caution, as your ability to grip is also heavily impacted.
- Persons requiring supervision due to their age or mental/physical impairment must be supervised during every application of the product.
- Avoid contact of the product with fatty and acidic agents, ointments, lotions, bases, sand or salt water.
- If hypersensitivity has been observed with you, consult your physician before the first application of the product. If hypersensitivity/ an allergic reaction occurs in relation with the application of the product, pause the application and consult a physician immediately.
- Never expose your product to direct sunlight, heat or cold for longer periods of time.

Contains natural rubber latex (elastodiene) which may cause skin irritations if hypersensitivity is present. Doff the product if you notice any allergic reactions.

Application instructions

- Observe the instructions for donning and doffing of the product as well as any additional instructions for use given by your physician/professional¹ under any circumstances.
 - Since the product unfolds its effects especially during physical activity, doff it during longer rest phases.
 - Check the product for integrity before every use. Only intact products may be used.
 - Make sure to have your physician/professional¹ show you how to don/doff the product and explain to you how it works until no doubts remain. If the affected part of the body changes in diameter, the product may have to be readjusted/newly prescribed.
- ### Donning of the product
- Undo the band and place the EpiPoint strap on the forearm, ensuring the five-point pad rests directly on the muscle belly, i.e. in lateral position (outside) **1b** in case of tennis elbow or in medial position (inside) **1b** in case of golfer's elbow. The rigid part of the strap should always face outwards.
 - To fasten the strap, rest the forearm on a firm surface (table, etc.) and tighten the Velcro fastening **2a** **2b**. Please avoid fastening the orthosis too tight! If the red warning section indicates excessive contact pressure, make sure to loosen the strap accordingly.

Cleaning instructions

To avoid damage and to preserve their long-term adhesive strength, we recommend closing the Velcro® fastenings before washing.

Observe the supplementary instructions on the sewn-in label/sticker on your product. Wash the product at 30 °C/86 °F.

Regular care ensures the ideal effectiveness of the product.

Area of use

Elbow joint

Notes on reuse

The product is intended for the individual treatment of one patient only.

Warranty

The legal provisions of the country in which the product was purchased apply. If you suspect a warranty claim, please first contact the person from whom you purchased the product directly. The product must be cleaned before making a warranty claim. If the instructions for handling and caring for the product have not been observed, the warranty may also be affected or excluded.

The warranty does not cover:

- Off-label use
- Failure to observe the instructions of the physician/professional¹
- Arbitrary product modifications

Disclaimer

In general, do not self-diagnose, self-treat or self-medicate unless you are a professional¹. Before first using our medical device, actively consult a physician or professional¹ since this is the only way to assess the individual effect of the product on your body and determine any possible risks of application/side effects. Please observe the advice of this professional¹ as well as all notes and instructions in this document or its online version (including texts, images, graphics, etc.), including any excerpts.

Obligation to report

Due to regional legal regulations, you are obliged to report any serious incidents involving the use of this medical device to both the manufacturer and the competent authority for medical devices (in Germany: Bundesinstitut für Arzneimittel und Medizinprodukte

- Federal Institute for Drugs and Medical Devices).

Disposal

Please dispose of the product in accordance with local regulations.

Material content

Thermoplastic polyurethane (TPU), Polyamide (PA), Thermoplastic elastomer based on styrene-ethylene-butylene-styrene (TPS-SEBS), Polyurethane (PUR), Polyoxymethylene (POM), Cotton (CO), Polyester (PES), Spring steel (FE), Elastodiene (ED), Silicone (SI), Elastane (EL)

MD – Medical Device

UDI – DataMatrix identifier as UDI

Date of information: 2026-02

- 1 A professional is any person who is authorized to adjust supports and orthoses and instruct others in their use in accordance with the applicable state and institutional regulations and other guidelines. As the manufacturer, we strongly recommend that this person disposes of orthopedic technical training or comparable acquired skills, as well as orthopedic technical orthopractice.
- 2 Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso

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Bauerfeind Ibérica, S.A.
Bauerfeind AG
Bauerfeind Middle East FZ LLC
Bauerfeind U.S. Inc.