AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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Dear patient,

Thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your physician or medical retailer.

Intended purpose

MalleoLoc L is a medical device. It is an orthosis¹ for the lateral stabilization of the ankle.

⚠ During the first fitting of the MalleoLoc L, the orthosis must be individually adapted to the patient by a professional². This is the only way to guarantee the full effect and ideal wearing comfort of the orthosis.

Indications

- Non-surgical treatment of sprains and ruptured ligaments
- Acute and chronic capsular ligament instabilities
- Prevention of chronic ligament instability
- Post-operative rehabilitation / protection, e.g. after suture repair / ligament reconstruction
- Post-traumatic / post-operative irritation

Risks of using this product

⚠ Caution³

Please closely observe the specifications in these instructions for use and the instructions given by the professional.

- The MalleoLoc L must only be worn in accordance with the specifications contained in these instructions for use, and the areas of application listed.
- The use in combination with other products, e.g. when wearing compression stockings as part of compression therapy, must be discussed beforehand with your physician.
- No product liability is accepted in the event of improper use.
- Do not make any alterations to the product. Failure to comply with this requirement may adversely affect product performance, thus canceling any product liability.
- Side effects involving the body as a whole have not been reported to date. Correct usage / donning of the product is assumed.
- Do not don the MalleoLoc L too tightly, as this could lead to local pressure symptoms. In rare cases, constriction of blood vessels and nerves may occur. If this happens, loosen the straps of the MalleoLoc L.
- If you notice any unusual changes or your symptoms get worse, stop using the MalleoLoc L and please contact your physician without delay.
- Caution: When further loosening the straps or removing the product, there is not sufficient support for the injury. In this case, avoid any stress on the affected leg or foot.
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a professional before first using the product. Restrictions may exist given the specific indication which the professional can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.
- Furthermore, proceed with extreme caution, as your ability to stand and walk is also heavily impacted.
- Check the specific ground characteristics and ensure the ground is not slippery whenever walking, standing or otherwise stressing the affected part of the body. If the ground is smooth, there is an acute risk of slipping and thus injury!

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- Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids.
- Please note: Never expose the MalleoLoc L to direct heat! This may damage the material, which can impair the effectiveness of the MalleoLoc L.

Contraindications

No clinically significant hypersensitivity reactions have been reported to date. If you have any of the following conditions, such aids should only be donned and worn after consultation with your physician:

- Skin disorders / injuries in the part of the body being treated, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up
- Impaired sensation and movement disorders of the legs / feet (e.g. diabetes mellitus)
- Impaired lymph drainage, including soft tissue swellings of uncertain origin in parts of the body where the orthosis is not worn

Application instructions

Donning of the MalleoLoc L

We recommend wearing hosiery under the MalleoLoc L.

Please check the floor and other walking surfaces to make sure they are non-slip before taking your first steps. Only run with extreme caution using short steps, remaining conscious of your ankle injury. Avoid slipping, as this could cause serious secondary injuries.

1. Sit down to don the MalleoLoc L.
2. Loosen the upper and lower strap. The lower strap remains threaded into the slot on the sole. Loosen the lower strap so far that you can easily put your foot through the loop you created. **1**
3. Place your foot on the floor at a 90° angle to your lower leg, as this is the best position for donning the orthosis. Now slide the affected foot into the orthosis between the strap loop and the shell, so that the shell sits laterally on the foot **in front** of the lateral malleolus.
4. Using one hand, press the shell onto the foot so that it sits flat against it. **2** Use the other hand to guide the lower strap diagonally across the instep by pulling from the outside of the foot (from the shell). Fasten the strap at the lower Velcro triangle **3**. During this process, the side of the strap with the blue velour must be attached to the blue plastic shell.
5. Now wind the upper strap around the base of the calf by pulling the strap slightly, **4** then fasten it. **5**
6. Now guide the lower strap above the medial malleolus and around the back of the Achilles tendon, **6** returning it to the front of the foot of the affected ankle. Fasten the end of the strap on the strap (the Velcro area depends on the calf circumference). **7**
7. Tighten both straps again, if necessary. **8** In the event of increased sensitivity on the sole of the foot, you have the option of removing the Velcro pad.

Doffing of the MalleoLoc L

1. Open the two Velcro straps.
2. Remove your foot from the orthosis.

Cleaning instructions

The sewn-in label of the MalleoLoc L, which contains information on the product name, manufacturer, washing instructions, and CE mark, is located on the inside of the upper strap.

To clean the MalleoLoc L, open the Velcro fastenings. Fasten the ends back onto the straps to avoid any damage. Use a damp cloth to wipe the shell. To clean the cushion of the MalleoLoc L, remove it from the shell's Velcro. You can wash it separately by hand with mild detergent at 30 °C.

Part of the body this product is used for

In accordance with the indications (ankle).

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Assembly and fitting instructions

The MalleoLoc L is supplied ready for use.

Assembling the MalleoLoc L after cleaning

Fitting the cushion

Attach the cushion to the plastic shell using each Velcro fastening point. Start with the sole. Make sure the cushion always protrudes slightly beyond the shell, and there are no folds.

Fitting the straps

The example below is for fitting the straps for the right side MalleoLoc L.

Place the plastic shell in front of you with the sole pointing toward you, and the outside pointing up (cushion facing down). Take the upper, shorter strap and position it in front of you with the blue velour pointing up. Thread the transparent Velcro end into the right-hand slot of the upper three slots, and back to the outside through the slot immediately adjacent. Attach it to the strap. Now take the other strap end, guide it toward the left in the direction of the left slot, and push it through the plastic shell from the outside.

Take the lower, longer strap and position it in front of you with the blue velour pointing down. Insert the transparent Velcro end in the slot in the middle of the orthosis from the right-hand side and from above, and attach it to the strap on the right. Take the other strap end and insert it into the opening in the sole.

Technical specifications / parameters

The MalleoLoc L is an ankle orthosis with two straps attached to it. The straps and the cushion can be removed.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. The warranty may be limited or excluded if the instructions on how to handle and care for the MalleoLoc L have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the professional were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a professional¹. Before first using our medical device, please seek the advice of a physician or professional² as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this professional as well as all instructions in this document³ or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the professional, please contact your physician or distributor, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority.

Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyketone (PK), Polyamide (PA), Polyurethane (PUR), Polyester (PES), Cotton (CO), Elastane (EL)

 – Medical device

 – DataMatrix Unique Device Identifier UDI

Date of information: 2025-03

¹ Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso.

² A professional is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

³ Note on the risk of personal injury (risk of injury, harm to health, and accidents) or damage to property (damage to the product).

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