**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0) 800 4430-130  
F +43 (0) 800 4430-131  
E info@bauerfeind.at

**BENELUX**

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**ManuLoc:**  
Aluminium (Al), Polyamid (PA), Polyurethan (PUR), Polyether Polyoxymethylen (POM), Polyester (PES)

**ManuLoc long:**  
Polyamid (PA), Aluminium (Al), Polyurethan (PUR), Polyether, Polyoxymethylen (POM), Polyester (PES)

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 english

## Dear patient,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

### Intended purpose

ManuLoc / ManuLoc long are medical devices. ① ManuLoc is an orthosis<sup>1</sup> for immobilizing the wrist. ② ManuLoc long is an orthosis<sup>1</sup> for immobilizing the wrist and parts of the forearm.

**If the product is not used properly, all warranty rights will be rendered null and void.**

### Indications / Areas of Application ManuLoc

- Post-traumatic or post-operative irritation
- Pre-operative care
- Carpal tunnel syndrome (mild)
- Tenosynovitis (moderate)
- Sprains / contusions
- Osteoarthritis (moderate)

### ManuLoc long

- Post-traumatic or post-operative irritation, e.g. following treatment with a plaster cast in the case of radius fractures
- Pre-operative care
- Carpal tunnel syndrome (moderate)
- Tenosynovitis (moderate, with pain while resting and at night)
- Sprains / contusions
- Osteoarthritis (moderate)

### Risks of using this product

#### Caution<sup>2</sup>

**Please closely observe the specifications in these instructions for use and the instructions given by the specialist<sup>3</sup>.**

- ManuLoc / ManuLoc long must only be worn in accordance with the specifications contained in these instructions for use and used for the areas of application (intended use, parts of the body) listed.
- To ensure the optimum fit of the ManuLoc / ManuLoc long, the required size of ManuLoc / ManuLoc long must be determined by a professional<sup>3</sup>.
- The sewn-in label of the ManuLoc / ManuLoc long, which contains information on

the size, manufacturer, washing instructions and CE mark, is located in the insertion opening for the wide stay located on the inside of the hand (the volar stay). Only a professional<sup>3</sup> may carry out the (initial) fitting of the product and provide instruction.

- Use in combination with other products, e.g. as part of compression treatment, must be discussed beforehand with your physician.
- No product liability is accepted in the event of improper use. Do not allow the product to come into contact with ointments, lotions or substances containing grease or acids.
- No side effects that affect the entire body have been reported to date. Correct usage / fitting is assumed. Any medical aids – such as supports and orthoses<sup>1</sup> – applied externally to the body can, if tightened excessively, lead to local pressure symptoms or, rarely, constrict the underlying blood vessels or nerves.
- Should you notice any unusual changes or experience discomfort, loosen the straps or take the product off and please contact a physician without delay.
- Caution: When loosening the straps or tensioning and fastening systems, or when removing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a medical professional before first using the product. Restrictions may exist given the specific indication which the professional<sup>3</sup> can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.

### Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Impaired sensation and circulatory disorders of the arm / hand, e.g. in the case of diabetes mellitus.
- Impaired lymph drainage – including soft tissue swellings of uncertain origin located away from the fitted aid.

### Application instructions

#### Putting on the ManuLoc / ManuLoc long

① Start by loosening the three Velcro straps on the lower arm splint (or the four Velcro straps on the long version). ② Place the ManuLoc / ManuLoc long on the correct hand. The straps are thus located on the top of the lower arm and on the back of the hand. ③ Fasten the straps on the lower arm splint in the following sequence: Lower arm straps 1, 2 and the strap over the back of the hand (or lower arm straps 1, 2, 3 and the strap over the back of the hand on the long version).

#### Taking off the ManuLoc / ManuLoc long

Loosen the straps on the lower arm splint in the reverse order. Take off the ManuLoc / ManuLoc long.

### Cleaning instructions

Please note the following:

- Remove the Velcro straps if possible, or fasten them securely in position.
- Air dry the product. Do not dry it in a tumble-dryer because this can damage the product.
- Please only wash the orthosis<sup>1</sup> by hand at 30 °C using a mild detergent. All components can stay on the orthosis<sup>1</sup> during washing. Leave the orthosis<sup>1</sup> to air dry.
- Regular care guarantees optimum effectiveness of the product. Also observe the instructions on the sewn-in label on the upper edge of your product.
- Never expose the product to direct heat as this may impair its effectiveness. If you have any complaints, please contact your retailer. Only clean goods are handled.

## Part of the body this product is used for

In accordance with the indications (wrist, lower arm). See Intended use.

## Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

## Assembly and fitting instructions

The ManuLoc / ManuLoc long is supplied in an anatomically contoured state. The orthosis is equally suitable for both the right and left hand.

## Technical specifications / parameters, accessories

The ManuLoc / ManuLoc long is a stabilizing orthosis for the wrist. It consists of a long lower arm splint. The straps on the lower arm splint are detachable.

Spare part ManuLoc long – spare straps (x4), color: titan 129 5343 0080 000

## Notes on reuse

This product is intended to be fitted to one patient only.

## Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the ManuLoc / ManuLoc long have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

## Disclaimer

In general, do not diagnose or treat yourself or decide on which medication to take unless you are a professional<sup>3</sup>.

Before first using our medical device, please seek the advice of a physician or professional<sup>3</sup> as this is the only way to assess the individual effect of the product on your body and determine possible risks / side effects of using this product. Follow the advice of this professional<sup>3</sup> as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics, etc.).

If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

## Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure. Our contact details can be found on the back of this brochure.

## Disposal

Once you have finished using the product, please dispose of it according to local specifications.

## Material content

### ManuLoc:

Aluminum (Al), Polyamide (PA), Polyurethane (PUR), polyether, Polyoxymethylene (POM), Polyester (PES)

### ManuLoc long:

Polyamide (PA), Aluminum (Al), Polyurethane (PUR), polyether, Polyoxymethylene (POM), Polyester (PES)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Version: 2025-12

## Specialists<sup>3</sup>

### General instructions

- Check the optimal and individual fit of the ManuLoc / ManuLoc long at the first fitting, and adjust if necessary.
- Practice the correct procedure for putting on the support with the patient.
- No improper modifications may be made to the product. Failure to comply with this requirement may adversely affect the medical effectiveness of the product, so that product liability is excluded.
- The ManuLoc / ManuLoc long is intended for the treatment of one patient only.

The aluminum stays are anatomically contoured, but they can still be individually adjusted to the patient.

### Shaping the stays:

- 1 Carefully remove the stays from the insertion pockets and shape them to the anatomy of the patient. Reinsert the three stays into the pockets provided.

<sup>1</sup> Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control or correct the limbs or the torso

<sup>2</sup> Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the product)

<sup>3</sup> A specialist is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

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