

 **BAUERFEIND AG**
Triebeser Straße 16
07937 Zeulenroda-Triebes
Germany
P +49 (0) 36628 66-40 00
F +49 (0) 36628 66-44 99
E info@bauerfeind.com
BAUERFEIND.COM


RhizoLoc®

Gebrauchsanweisung
Instructions for use
Notice d'utilisation
Istruzioni per l'uso
Instrucciones de uso


 **MADE**
 **IN**
 **GERMANY**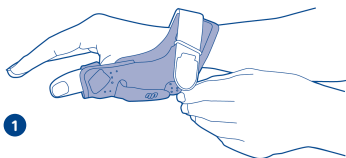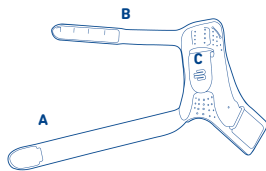

1

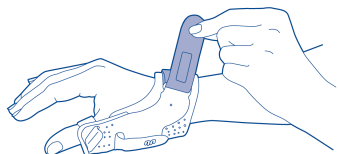2

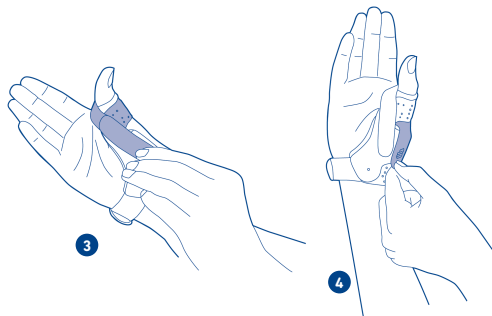3

4

(de)	deutsch.....	3	(sr)	srpski.....	54
(en)	english.....	6	(sl)	slovenski.....	57
(fr)	français.....	10	(ro)	romanian.....	60
(nl)	nederlands.....	13	(el)	ελληνικά.....	63
(it)	italiano.....	16	(tr)	türkçe.....	67
(es)	español.....	20	(ru)	русский.....	70
(pt)	português.....	23	(ee)	eesti.....	74
(sv)	svenska.....	26	(lv)	latviešu.....	76
(no)	norsk.....	29	(lt)	lietuvių.....	80
(fi)	suomi.....	32	(uk)	українська.....	82
(da)	dansk.....	35	(kk)	қазақ.....	86
(pl)	polski.....	38	(he)	לְשׂוֹנָה.....	90
(cs)	česky.....	42	(ja)	日本語.....	93
(sk)	slovensky.....	45	(ko)	한국어.....	96
(hu)	magyar.....	48	(ar)	عربي.....	99
(hr)	hrvatski.....	51	(zh)	中文.....	102

(de) deutsch

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Polyurethan (PUR), Polyamid (PA), Aluminium (Al), Polyether, Polyester (PES), Polyoxymethylen (POM)

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 english

Dear patient,

Thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products

every day – because your health is very important to us. Please read and observe these instructions for use carefully. If you have any questions, please contact your physician or medical retailer.

Intended purpose

RhizoLoc is a medical device. It's an orthosis used for the stabilization of the metacarpophalangeal and carpometacarpal joints of the thumb.

Indications

- Post-operative and post-traumatic irritation
- Collateral ligament injuries, e.g. skier's thumb
- Osteoarthritis of the first carpometacarpal joint (midl)
- Sprains / contusions

Risks of using this product

 **Caution***

Please closely observe the specifications in these instructions for use and the instructions given by the professional.

RhizoLoc must only be worn in accordance with the specifications contained in these instructions for use and used for the areas of application (places of use) listed. To guarantee the best possible fit of the RhizoLoc, it is essential to check whether the product is adequately shaped and whether it needs to be anatomically reshaped by a professional². Only a professional may carry out the (initial) fitting of the product and provide instruction.

The sewn-in label of the RhizoLoc, which contains information on the product name, size, manufacturer, washing instructions and CE certification, is located on the outside of the wrist strap.

Use in combination with other products, e.g. as part of compression treatment (e.g.: arm sleeves), must be discussed beforehand with your physician. No product liability is accepted in the event of improper use.

Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids. Do not make any alterations to the product. Failure to comply with this requirement may adversely affect product performance, thereby voiding any product liability.

Side effects involving the body as a whole have not been reported to date. Correct usage / donning of the product is assumed.

Do not don the RhizoLoc too tightly as this could lead to local pressure symptoms. In rare cases, constriction of blood vessels and nerves may occur. If this happens, loosen the straps of the RhizoLoc and, if necessary, check the size of your RhizoLoc.

If you notice any negative changes or an increase in symptoms while using the product, stop any further use and contact your physician.

Caution: When loosening the straps or tensioning and fastening systems, or when doffing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.

If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a professional before first using the product. Restrictions may exist given the specific indication which the professional can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.

For persons who require supervision due to their age and/ or mental/physical impairments or who are unable to correctly don and adjust the orthosis independently, every use of the

product must be carried out under supervision or with assistance.

Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be donned and worn after consultation with your physician:

- Skin disorders / injuries in the part of the body being treated, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up
- Impaired sensation and circulatory disorders in the arm / hand, e.g. in cases of diabetes mellitus
- Impaired lymph drainage, including soft tissue swellings of uncertain origin in parts of the body where the orthosis is not worn

Application instructions

Donning of the RhizoLoc

- A - wrist strap,**
- B - thumb strap,**
- C - thumb tab**

① Open the two Velcro straps on the RhizoLoc. Loosen the straps without pulling them out of the guide. ② Position the RhizoLoc on the affected thumb. To do this, guide your hand through the bottom wrist strap and your thumb through the top thumb strap. ③ Close the wrist strap. Then close the thumb strap by attaching the Velcro at the side next to the thumb tab.

④ Following consultation with your physician, you can use the thumb tab to adjust the level of stabilization of the thumb basal joint in accordance with the phase of your treatment.

Doffing of the RhizoLoc

Open the thumb and wrist straps. Guide your hand out of the RhizoLoc.

Cleaning instructions

Please note the following:

- Remove the Velcro straps if possible, or fasten them securely in position.
- Air dry the product. Do not dry it in a tumble-dryer because this can damage the product.
- Please only wash the orthosis¹ by hand at 30 °C using a mild detergent. All components can stay on the orthosis¹ during washing. Leave the orthosis¹ to air dry.
- Regular care guarantees the ideal effectiveness of the product. Also observe the instructions on the sewn-in label on the upper edge of your product.
- Never expose the product to direct heat as this may impair its effectiveness. If you have any complaints, please contact your distributor. Only clean goods are handled.

Part of the body this product is used for

In accordance with the indications (thumb). Please see intended purpose.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Assembly and fitting instructions

The RhizoLoc is supplied in an anatomically contoured state. If necessary, the RhizoLoc can be contoured further. The orthosis is available in right- and left-handed designs.

Technical specifications / parameters

The RhizoLoc is an orthosis for the thumb. No spare parts are available.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product

directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. The warranty may be limited or excluded if the instructions on how to handle and care for the RhizoLoc have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the professional were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a professional. Before first using our medical device, please seek the advice of a physician or professional, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this professional as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the professional, please contact your physician or distributor, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyurethane (PUR), Polyamide (PA), Aluminum (Al), polyether, Polyester (PES), Polyoxymethylene (POM)

MD – Medical device
UDI – DataMatrix Unique Device Identifier UDI

Date of information: 2026-05

Professionals²

General instructions

- Check the ideal and individual fit of the RhizoLoc after the first fitting. Make any corrections if necessary.
- Practice the correct procedure for donning of the product with the patient.
- No improper modifications may be made to the product. Failure to comply may adversely affect the product's performance, thus canceling any product liability.
- The RhizoLoc is intended for the treatment of one patient only. The aluminum frame is anatomically contoured, but needs to be fitted individually for each patient. To do this, apply a little force to form the aluminum frame into the anatomically correct shape.

The thumb tab to limit the thumb basal joint's range of motion must be firmly strapped in and under tension when treatment begins and must contain no folds. It is the physician's responsibility to adjust the level of restriction of movement of the thumb.

* Note on the risk of personal injury (risk of injury, harm to health, and accidents) or damage to property (damage to the product).

¹ Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso.

² A professional is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

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Bauerfeind Ibérica, S.A.
Bauerfeind AG
Bauerfeind Middle East FZ LLC
Bauerfeind USA, Inc.