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Polyamid (PA), Polyurethan (PUR), Polyoxymethylen (POM), Polyester (PES), Polypropylen (PP)

Barrierefreie Version

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Stand der Information: 2023-08

Dear Customer,

thank you for choosing a product from Bauerfeind. This shoulder strap is a supplementary component for use with our OmoLoc and SecuTec Omo orthoses. **In addition to these brief instructions, please also read through the instructions for use for the OmoLoc or SecuTec Omo carefully and in their entirety.** Please keep these instructions for use safe, as you may want to read them again later. If you have any questions, please contact your physician or medical retailer.

The shoulder strap can be used with the products OmoLoc and SecuTec Omo to provide additional stabilization, if required.

If necessary, ask someone else to help you attach the shoulder strap and put on the OmoLoc or SecuTec Omo orthosis.

Attaching the shoulder strap to the OmoLoc orthosis

1 Use the Velcro fastening to attach the shoulder strap to the outside of the OmoLoc orthosis in the back area. Put the orthosis on as described in the OmoLoc instructions for use. **2** Guide the strap over the healthy shoulder to the front. **3** Using the square Velcro tab, fasten the end of the shoulder strap either onto the forearm strap or **4** onto the abdominal strap. **5** Now undo the round Velcro end and adjust the length of the strap to fit.

Attaching the shoulder strap to the SecuTec Omo orthosis

1 Use the Velcro fastening to attach the shoulder strap to the outside of the SecuTec Omo orthosis in the back area. Put the orthosis on as described in the SecuTec Omo instructions for use. **11** Guide the strap over the healthy shoulder to the front. Using the square Velcro tab, fasten the end of the shoulder strap onto the abdominal strap. Now undo the round Velcro end and adjust the length of the strap to fit.

Cleaning instructions

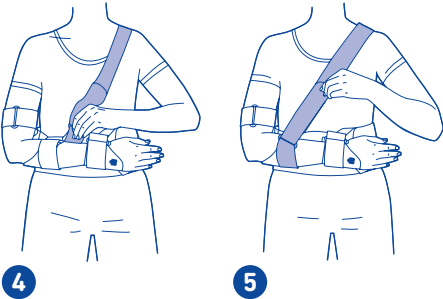
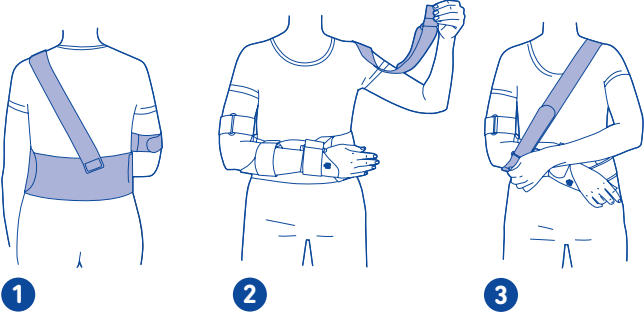
Note: Never expose the shoulder strap to direct heat (e.g. heaters, sunlight, in your car, etc.)! This may cause damage to the material, which can impair the function of the strap. Close the Velcro fastenings to avoid any damage. Wash the strap by hand using a mild detergent. Then leave the shoulder strap to air dry.

Material content

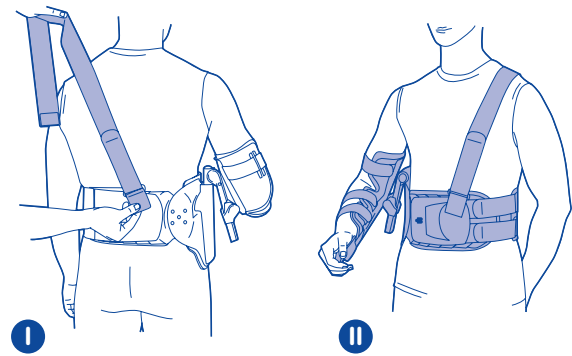
Polyamide (PA), Polyurethane (PUR), Polyoxymethylene (POM), Polyester (PES), Polypropylene (PP)

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