AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear patient,

Thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your physician or medical retailer.

Intended purpose

SofTec Lumbo is a medical device. It is an orthosis¹ for immobilizing the lumbar spine with mobilization function with removable shell.

Indications

- Spondylolysis / spondylolisthesis, grades II–III

- Most severe lumbosciatica with muscular deficiencies in cases of significant intervertebral disk protrusions / herniated disks
- Extremely severe (pseudo) radicular lumbar syndrome
- Degenerative facet syndrome
- E.g. prolapse, e.g. post-operative
- After a discectomy
- E.g. spondylolysis / kyphoplasty (one or more levels)
- Lumbar spinal canal stenosis
- Vertebral fractures (lumbar spine) with vertebral body damage
- Tumors (metastases)
- Severe degeneration / muscular insufficiency

Risks of using this product

⚠ Important information

Please closely observe the specifications in these instructions for use and the information provided by professionals².

- SofTec Lumbo is a prescribed product that should be applied under a physician's guidance. The orthosis must be applied in accordance with the information contained in these instructions for use and for the listed indications.
- The plastic shell or stays of the SofTec Lumbo may only be modified by a professional². Failure to comply with this requirement may adversely affect product performance, thereby canceling any product liability.
- If the knitted sections of the SofTec Lumbo are subsequently altered, the performance of the orthosis can no longer be guaranteed.
- Please consult your physician before using SofTec Lumbo in combination with other products.
- Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids.
- For information regarding the options available for returning the packaging to ensure it is correctly recycled, please observe the instructions from the sales outlet where you purchased this product.
- Side effects involving the body as a whole have not been reported to date. Correct usage / donning of the product is assumed. Any medical aids – supports and orthoses – applied externally to the body can, if tightened excessively, lead to local pressure symptoms or, rarely, also constrict underlying blood vessels or nerves. In such cases, please loosen the straps or the fastening slightly and have the correct fit and size of the orthosis checked if necessary.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your physician.
- Caution: When loosening the straps or tensioning and fastening systems widely, or when doffing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.

 Contains natural rubber latex (elastodiene), which can cause skin irritation in persons with an allergy. Doff the orthosis if you notice any allergic reaction.

Contraindications

- If you have any of the following conditions, such aids should only be donned and worn after consultation with your physician:
- Skin disorders / injuries to the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up
- Impaired sensation
- Severe restriction of the performance of the heart and lungs (risk of a rise in blood pressure if the aid is worn during intense physical exertion)

Application instructions

Donning and doffing of the SofTec Lumbo

Despite its special design, the SofTec Lumbo is donned and doffed in the same way in all stages.

Donning:

- The shell individually adapted by a professional² must sit loosely on the iliac crests in the pelvic area (Fig. 2). When donning the orthosis without the shell, it must be ensured that the support is positioned correctly. If necessary, practice correct positioning with the professional².
- Wrap the orthosis around your body in such a way that the reinforcements visible on the outer knitted fabric are positioned parallel to the spine (Fig. 3); sewn-in labels at the top.

- To don the inner knitted fabric, place your fingers in the loops provided on the two light-gray fastening halves and pull the fastener forward evenly (Fig. 5).
- Press the left half of the fastener against the abdomen and pull the right half over it to secure the Velcro fastening. When inserting the pressure cushion for the abdominal wall, ensure correct positioning (the pressure cushion should be placed centrally underneath the fastener).
- When fastening the support, remove your fingers from the left hand strap first and then from the right hand strap (Fig. 6).
- Take the two LPT (**low power tensioning**) straps in both hands and pull them forward (Fig. 7). Then attach the two LPT straps to the large fastening using the Velcro (Fig. 8). Please note: With the “Long” version, always fasten the lower straps first, then fasten the upper straps using the Velcro.
- When first donning the orthosis, the optimal, individual fit must be checked, and corrected if necessary, by a professional². Correct donning should be practiced.

Doffing:

- Unfasten the LPT straps of the outer knits and pull them back so that the orthosis fits loosely; then fasten the ends of the straps on the loosened parts.
- Open the large, light-gray fastener.
- Doff the orthosis and refasten the light-gray fastener.

Stage 1 (Immobilization phase):

Securing the spine after surgery or at the start of non-surgical treatment by using a shell.

Stage 2 (Mobilization phase):

To allow for mobilization over the course of recovery, the shell is removed and replaced by CFRP stays (carbon fiber reinforced plastic rods) in the knitted fabric.

Stage 3 (Activity phase):

With increasing activity, the CFRP stays are replaced by corset stays, thereby reducing the external supportive effect.

⚠ SofTec Lumbo must be individually adapted to the patient by a professional². This is the only way the full effectiveness¹ and ideal wearing comfort of the orthosis are guaranteed. If the product is not used properly, any claim for warranty will expire.

Cleaning instructions

⚠ Never expose the orthosis¹ to direct heat (e.g. heater, sunlight, etc.)!

SofTec Lumbo can be hand-washed separately using a mild detergent at 30° C. The plastic shell must be removed first. To do this, proceed as follows:

- Unfasten the two thin elastic straps. Pull these straps out of the eyelets to remove them.
- Unfasten the two Velcro patches that connect the outer and the inner knitted fabric at the center of the back.
- Remove the plastic shell from the orthosis.
- The plastic shell can easily be cleaned with a sponge using a pH-neutral soap or a shampoo. Do not dry the product using external heat (hairdryer, radiator), but use a towel or air-dry it.
- Removing the CFRP or corset stays is not necessary.
- If the stays of the outer knitted fabric are to be removed, bend the fasteners in the upper area (red area) and pull the stays out (Fig. 9).
- To prevent damage and preserve their long-term adhesive strength, we recommend closing the Velcro fastenings before washing (including the Velcro patches between the inner and outer knitted fabric).
- Please follow the steps in reverse order after washing the product. When reinserting the CFRP stays, pay attention to the markings to find the correct orientation (top / bottom, left / right). Ensure the knitted fabric is positioned correctly on the plastic shell. The Velcro patches are color-coded. The sewn-in labels are always on top, the plastic shell has a marking in the upper area.

This product has been tested by our own quality control system. Nevertheless, if you have a complaint, please contact your medical retailer.

Notes on reuse

This product is intended to treat one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. The warranty may be limited or excluded if the instructions on how to handle and care for the SofTec Lumbo have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the professional² were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a professional². Before first using our medical device, please seek the advice of a physician or professional² as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this professional² as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics, etc.). If you still have doubts after your consultation with the professional², please contact your physician or distributor, or contact us directly.

Duty to report

Because of regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority (in Germany: Bundesinstitut für Arzneimittel und Medizinprodukte - Federal Institute for Drugs and Medical Devices). Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyethylen (PE), Polyamide (PA), Polyurethan (PUR), stainless steel, Elastodiene (ED), Ethylene propylene diene monomer rubber (EPDM), Cotton (CO), Polyoxymethylene (POM), Elastane (EL), Polyester (PES), Glass fiber (GF), Thermoplastic polyurethane (TPU), Carbon (CF)

MD – Medical device

UDI – DataMatrix Unique Device Identifier UDI

Date of information: 2025-09

¹Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso.

²A professional is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

Professionals²:

Application instructions

Donning and doffing of the SofTec Lumbo

Despite its special design, the SofTec Lumbo is donned and doffed in the same way in all stages.

Donning:

- Wrap the orthosis around your body so that the inserts attached to the outer knitted fabric for the CFRP and corset stays are positioned parallel to the spine (Fig. 1), sewn-in label at the top.
- The anatomical individually adapted shell must be positioned by the professional² evenly above the iliac crests (Fig. 2).
- When fastening the inner knitted fabric (preferably done by the patient), ensure that the orthosis is centered (Fig. 3).
- The preformed pelvic indentations of the individually adapted shell must sit loosely on the iliac crests. When donning the orthosis without the shell, it must be ensured that the support is positioned correctly (upper label).
- To fit the shell individually, wedges can be partly or fully cut out or the shell can be adapted (Fig. 4).
- To don the inner knitted fabric, the patient must place their fingers in the loops provided on the two light-gray fastening halves and pull the fastening forward evenly (Fig. 5).

- The left half of the fastener must be pressed against the abdomen and the right half pulled over it to secure the Velcro fastening (Fig. 6). If necessary, ensure that the enclosed pressure cushion for the abdominal wall (this should be fastened centrally to the lower left half of the fastening using the Velcro).
- When fastening the support, the fingers must first be removed from the left loop first and then from the right loop.
- The patient must take the two LPT (**low power tensioning**) straps in both hands and pull them forward at the same time until the correct level of tension is created (Fig. 7). Then attach the two LPT straps to the large fastening using the Velcro (Fig. 8). With the “Long” version, always fasten the distal (lower) straps first, then fasten the proximal (upper) straps using the Velcro. The plastic buckles of the LPT straps can be adapted in their shape thermoplastically (shaped using heat) if necessary.
- Check the ideal individual fit and adapt if needed.
- The CFRP stays in the outer knitted fabric are thermoformable (using heat). The inserts are open on the top side so that the stays can be removed (or replaced with corset stays). When reinserting the CFRP stays, pay attention to the markings to find the correct orientation (top / bottom, left / right; Fig. 9).
- The patient should practice the correct way of donning the orthosis with help from the professional².

Doffing:

- Unfasten the LPT straps of the outer knits and pull them back so that the orthosis fits loosely; then fasten the ends of the straps on the loosened parts.
- Unfasten the two light-gray fastening halves.
- Doff the orthosis and refasten the light-gray fastener.

Adaptation of SofTec Lumbo

- The shell must be adapted to the patient’s individual anatomy. Pressure peaks must always be avoided. You can heat up the plastic shell to about 80 °C using a hot-air gun.
- You can also heat up and reshape the CFRP stays to fit the patient’s anatomy.

⚠ Please note: when using the SofTec Lumbo for treatment following fusion surgery (spondylodesis), the CFRP stays need to be adapted to the normal spinal column oscillation (anatomically correct lordosis angle) to avoid excessive straightening force (lordosis reduction).

- When the SofTec Lumbo is used to bridge lordosis, insert the CFRP stays in their original shape with the convex side facing outwards (instead of adapting them to the shape of the body as is done to stabilize lordosis).
- If bending hooks are employed, only round bending hooks should be used.
- You can find instructions on how to remove the shell and the stays under the keyword “Cleaning instructions”.
- The ideal temperature for adapting the stays is between 170 °C and 190 °C. Heating above these temperatures leads to destruction of the material.
- When heating up injection-molded parts, keep a minimum distance of 5 cm from the heat source to avoid burning the surface.
- The injection-molded part must be heated evenly over a large area (no spot heating!).

⚠ Excessive spot heating can damage the material. In such cases, no product liability can be accepted.

- The material should only be bent in large arches (no sharp bends).
- The cooling process can be accelerated with the aid of cold water without negatively affecting the material properties.
- You can heat and reshape the material more than once. The heating process does not visually alter the surface of the material. In order to find out the appropriate time to bend the material, test it during the heating process.
- Check the fit of the orthosis on the patient in both a standing and a sitting position after each modification.
- Local pressure peaks on the skin or in the area of the iliac crests must always be avoided.

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