AUSTRIA
Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

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Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended use

SofTec OA is a medical product. It is an orthosis¹ for the relief of medial osteoarthritis of the knee using adjustable air bladders.

Indication

- Medially affected osteoarthritis of the knee

Important information

- Modifications of SofTec OA or any joint bar adjustments may only be made by suitably trained specialists. Failure to comply with this requirement may adversely affect product performance, thereby cancelling any product liability.
 - **SofTec OA must be individually adapted to the patient by a trained orthopaedic technician in order to gain the maximum effect and optimal wear comfort from this orthosis¹.**
 - Please note: During airplane flights the air chambers must be deflated.
 - Should you notice any unusual changes (e.g. increased symptoms), please contact your medical provider without delay.
 - A combination with other products (e.g. compression stockings) must be discussed beforehand with your medical provider.
 - SofTec OA should be removed during prolonged rest periods (e.g. at night). During prolonged rest periods in the daytime (e.g. prolonged sitting), it is advisable to loosen the straps slightly. Before resuming activity, remember to check the correct fit / positioning of SofTec OA.
 - To avoid injury to the hand and fingers, do not place hands behind the thigh or knee when you crouch down or bend legs sharply. The joint shells can come together increasing the risk of injury to the hands and fingers.
 - Avoid the use of ointments, lotions or any substances including grease or acids.
 - Products with silicone adhesive bands can cause skin irritation in hypersensitive individuals.
 - Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed. Any supports and orthoses applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves. If you experience any abnormal sensations of the skin on the lower leg and / or foot or any weakness of the foot-raising muscles, remove the product immediately and consult your medical provider. Incorrect fitting or over inflating of the air pads may lead to irritation or injury to the peroneal nerve.
-  Contains natural rubber latex (elastodiene), which can cause skin irritation in persons with an allergy. Take off the orthose if you notice any allergic reaction.

Contraindications

- Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:
1. Post-thrombotic syndrome
 2. If the peroneal nerve has been surgically moved.
 3. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present.
Likewise, any raised scars with swelling, redness and excessive heat build-up.
 4. Varicose veins.
 5. Impaired sensation and circulatory disorders of the legs, e.g. diabetes mellitus.
 6. Impaired lymph drainage – including soft tissue swellings of uncertain origin located away from the fitted aid.

Application instructions

Important Notes for the Technicians

- The ideal temperature for molding is between 170 °C and 190 °C or 338 °F and 374 °F. Heating at higher temperatures will lead to the destruction of the material.
- When heating carbon / glass-fibres keep the material a minimum distance of 5 cm from the heat source to avoid burning the surface.
- The carbon / glass-fibre should be heated evenly across the surface to avoid damaging the material. In that case Bauerfeind will not be liable for the product.
- The material should only be bent gently and in large arches.
- The cooling process can be accelerated using cold water which will not change the material's characteristics.
- The material can be heated and molded repeatedly.
- The heating process does not visually alter the surface of the material. In order to find out the appropriate timefor bending, test it during the heating process.

Fitting SofTec OA

- Open both zip fasteners before fitting.
- Grasp SofTec OA with both hands at the top edge (at the label). Insert your index fingers in the two loops at the top edge to facilitate fitting.
- Rotate SofTec OA so that the back section is at the front. This also moves the lower zip fastener to the front (Figure 1).
- Pull the bottom edge of SofTec OA over the ankle.
- Rotate SofTec OA half a turn (180°) around the leg axis (Figure 2).
- With the knee flexed at 30°, pull SofTec OA up the leg until the kneecap is positioned between the contoured pads (Figure 3).
- Check the correct positioning of the kneecap which should, ideally, be exactly between the two marks (Figure 4).
- Close the zip fastener at the top of the thigh. This operation is facilitated by pulling together both parts of the zip cover (Figure 5).
- Then secure the cover itself.
- Next, close the lower zip fastener (Figure 6).
- Before fastening the straps, ensure that the joint bar is positioned on the outside of the leg, in line with the leg axis, when the leg is extended (Figure 7).

If you have decided to order the titanium-colored version, please proceed as follows:

- Pull the straps through the loops in the following sequence and fasten loosely:
 1. the two grey lower-leg straps, red plastic loops (Figure 8).
 2. the two grey thigh straps, red plastic loops (Figure 8).
 3. the red calf strap, metal loop (Figure 9).
- Now fasten the straps, pulling them uniformly, in the following sequence:
 1. simultaneously the two grey lower-leg straps (Figure 10).
 2. simultaneously the two grey thigh straps (Figure 11).
 3. the red calf strap (Figure 12).
- Now gently pull and fasten the red correction straps at the front of the lower leg (Figure 13).
- Fasten the two red correction straps at the thigh, pulling sideways uniformly on the Velcro® surfaces. Fasten the strap at the back first, followed by the front strap (Figures 14 / 15).

If you have decided to order the black version, please proceed as follows:

- Pull the straps through the loops in the following sequence and fasten loosely:
 1. the two black lower-leg straps, black plastic loops (Figure 8).
 2. the two black thigh straps, black plastic loops (Figure 8).
 3. the blue calf strap, metal loop (Figure 9).
- Now fasten the straps, pulling them uniformly, in the following sequence:
 1. simultaneously the two black lower-leg straps (Figure 10).
 2. simultaneously the two black thigh straps (Figure 11).
 3. the blue calf strap (Figure 12).
- Now gently pull and fasten the blue correction straps at the front of the lower leg (Figure 13).
- Fasten the two blue correction straps at the thigh, pulling sideways uniformly on the Velcro® surfaces. Fasten the strap at the back first, followed by the front strap (Figures 14 / 15).

- Open the flap at the top edge and inflate the integrated air pads using the supplied bulb pump (Figure 16).
- The amount of air can be adjusted individually such that symptoms are alleviated when walking. The pressure should not cause pain of discomfort, i.e. SofTec OA should still function correctly and feel comfortable to wear.

Removing SofTec OA

- Before removing SofTec OA deflate the air pads by gently squeezing on the sides of the valve (Figure 17).
- Undo the straps in the reverse sequence and take off SofTec OA by pushing it down over the foot. Fasten the straps by loosely closing the Velcro® fastenings.

Washing instructions and product care

- Never expose the orthosis to direct heat (e.g. heater, sunlight, etc.!). The sewn-in label of the the SofTec OA, containing information about the product name, size, manufacturer, washing instructions and CE identification, is located on the inside of the knitted fabric. Wash SofTec OA separately by hand at 20 – 30 °C using a mild detergent. First remove the joint bar with the air pad from the side pockets. To avoid damage and to preserve their long-term adhesive strength, we recommend closing the Velcro® fastenings before washing.
- Proceed as follows:
- First, undo the two thigh straps. Pull the straps out of the plastic loops. Turn the strap ends round and secure the Velcro® fastenings.
 - Repeat the procedure for the three lower-leg straps.
 - Open the three diagonal Velcro® straps and fasten the strap ends likewise by turning them round.
 - The side pockets can now be opened by pulling the whole Velcro® section to one side.
 - Remove the joint bar.
 - N.B.: Bear in mind that the joint bar is anatomically contoured and, after the washing process, should be reinserted, with the valve face upwards, in the same side pocket from which it was removed.
- Before washing, reinsert the straps through the plastic loops and close all Velcro® fastenings.

Part of the body this product is used for

In accordance with the indications (knee). See purpose.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Notes on reuse

The product is intended to be fitted only once and to one patient.

Warranty

- The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the SofTec OA have not been observed. Warranty is excluded if:
- The product was not used according to the indication
 - The instructions given by the specialist were not observed
 - The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

The product can be disposed of without any restrictions.

Version: 2019-12

¹ Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

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