**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0)800 4430-130  
F +43 (0)800 4430-131  
E [info@bauerfeind.at](mailto:info@bauerfeind.at)

**BENELUX**

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 english

## Dear patient,

Thank you for choosing a Bauerfeind product. We wish you a speedy recovery.

Please read and observe these instructions for use closely and in their entirety. If there is something you do not understand or if you have any questions, do not use the product until you have consulted your physician or professional<sup>1</sup> again. Keep these instructions for use in case you want to look something up later.

## Intended purpose

GenuPoint is a medical device. It is a support for the relief and stabilization of the patellar tendon.

## Indications

- Irritation in the patellar tendon area (patellofemoral pain syndrome, patellar chondropathy, patella tip syndrome / jumper's knee)

## Contraindications

For the following clinical pictures, usage of the product is only indicated upon consultation with your physician:

- Skin disorders/lesions on the part of the body to be treated, especially if inflammatory manifestations as well as raised scars with swelling, redness and overheating are present
- Varicose veins (varicosis)
- Impaired sensation and circulatory disorders of the legs, e.g. diabetes mellitus
- Impaired lymph drainage – including soft tissue swellings of uncertain origin located away from the fitted aid

**Ask your physician whether your conditions fall into these categories.**

## Notes before first usage

If you wish to use your Bauerfeind product for acute symptoms, injuries or healed previous conditions, you urgently need to obtain professional medical advice and observe it before using the product. Restrictions may apply to you

which you must take into account and which the professional<sup>1</sup> will clarify.

In case of identified or suspected problems with blood pressure and intense physical exercise, prolonged use of the product can have additional effects on your blood pressure.

After you have been prescribed/you have purchased the product, only use it according to the indication and under observance of any additional instructions given by your physician/professional<sup>1</sup>. If you wish to use the product in combination with medication, other medical devices or other treatments, consult your physician/professional<sup>1</sup> beforehand.

If any additional conditions or injuries occur during the period of application of the product, also consult your physician immediately.

### Risks of application

- Do not arbitrarily modify the product as this may cause it to not help as expected and/or worsen your health.
- The product must not be applied too firmly as this may cause local pressure symptoms or, in rare cases, constrict underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms related to the application of the product (such as tissue congestion, skin irritation, swelling, etc.), pause the application and consult your physician/professional<sup>1</sup>.
- If you completely doff the product, treatment is insufficient and additional damage to the part of the body affected may occur during additional strain. Be especially careful when this is the case.
- Additionally, restrictions due to the indication may apply which the professional<sup>1</sup> will explain to you. In particular, driving vehicles, other means of transportation or machines may only be possible under restrictions. The same applies to the use of ladders, tools and sports equipment. When in doubt, we recommend to suspend these activities.
- Persons requiring supervision due to their age or mental/physical impairment must be supervised during every application of the product.
- Avoid contact of the product with fatty and acidic agents, ointments, lotions, bases, sand or salt water.
- If hypersensitivity has been observed with you, consult your physician before the first application of the product. If hypersensitivity/

an allergic reaction occurs in relation with the application of the product, pause the application and consult a physician immediately.

- Never expose your product to direct sunlight, heat or cold for longer periods of time.

### Application instructions

- Observe the instructions for donning and doffing of the product as well as any additional instructions for use given by your physician/professional<sup>1</sup> under any circumstances.
- Since the product unfolds its effects especially during physical activity, doff it during longer rest phases.
- Check the product for integrity before every use. Only intact products may be used.
- Make sure to have your physician/professional<sup>1</sup> show you how to don/doff the product and explain to you how it works until no doubts remain. If the affected part of the body changes in diameter, the product may have to be readjusted/newly prescribed.

Donning of the product

- The GenuPoint can be worn on the right or left leg. Before donning it, please loosen both Velcro straps. Open one side completely by pulling the end out of the tab.
- Put the GenuPoint on in the center of the kneecap with the semicircular cut-out edge at the top **1**.
- Then thread the Velcro strap back through **2**.
- Now pull both strap ends at the same time with equal force to prevent the support from twisting. Make sure that the support is sitting correctly and is level with the patellar tendon **3**. You can adjust the pressure on the patellar tendon to suit you, but it must not be uncomfortable.
- Now fasten both Velcro ends to the strap **4**.
- Depending on your needs and activity level, you can readjust the individual pressure on the patellar tendon.

Doffing of the product

- To doff the support, first loosen both Velcro straps and then remove one side fully from the guide tab. For storage, both strap ends should be fastened again to avoid damage.

### Cleaning instructions

Observe the supplementary instructions on the sewn-in label/sticker on your product. Wash the product at 30 °C/86 °F. Regular care ensures the ideal effectiveness of the product. Fasten

both ends of the Velcro strap before washing to avoid damage.

### Area of use

Knee joint

### Assembly instructions

If your product has components that are hooked or laid into the product or attached using Velcro, check whether they are positioned as shown in the relevant product images.

### Notes on reuse

The product is intended for the individual treatment of one patient only.

### Warranty

The legal provisions of the country in which the product was purchased apply. If you suspect a warranty claim, please first contact the person from whom you purchased the product directly. The product must be cleaned before making a warranty claim. If the instructions for handling and caring for the product have not been observed, the warranty may also be affected or excluded.

The warranty does not cover:

- Off-label use
- Failure to observe the instructions of the physician/professional<sup>1</sup>
- Arbitrary product modifications

### Disclaimer

In general, do not self-diagnose, self-treat or self-medicate unless you are a professional<sup>1</sup>. Before first using our medical device, actively consult a physician or professional<sup>1</sup> since this is the only way to assess the individual effect of the product on your body and determine any possible risks of application/side effects. Please observe the advice of this professional<sup>1</sup> as well as all notes and instructions in this document or its online version (including texts, images, graphics, etc.), including any excerpts.

### Obligation to report

Due to regional legal regulations, you are obliged to report any serious incidents involving the use of this medical device to both the manufacturer and the competent authority for medical devices (in Germany: Bundesinstitut für Arzneimittel und Medizinprodukte - Federal Institute for Drugs and Medical Devices).

### Disposal

Please dispose of the product in accordance with local regulations.

### Material content

Polyamide (PA), Silicone (SI), Polyurethane (PU), Cotton (CO), Polyoxymethylene (POM), Polyester (PES), Elastane (EL), acrylic resin dispersion

 – Medical Device

 – DataMatrix identifier as UDI

### Notes for professionals<sup>1</sup>

- Check the optimal and individual fit of the GenuPoint after the first fitting. Make any corrections if necessary.
- Practice the correct procedure for donning of the product with the patient.
- No improper modifications may be made to the product. Failure to comply may adversely affect the product's performance, thus canceling any product liability.
- The GenuPoint is intended for the treatment of one patient only.

Date of information: 2026-02

<sup>1</sup> A professional is any person who is authorized to adjust supports and orthoses and instruct others in their use in accordance with the applicable state and institutional regulations and other guidelines. As the manufacturer, we strongly recommend that this person disposes of orthopedic technical training or comparable acquired skills, as well as orthopedic technical infrastructure.

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