GenuTrain®

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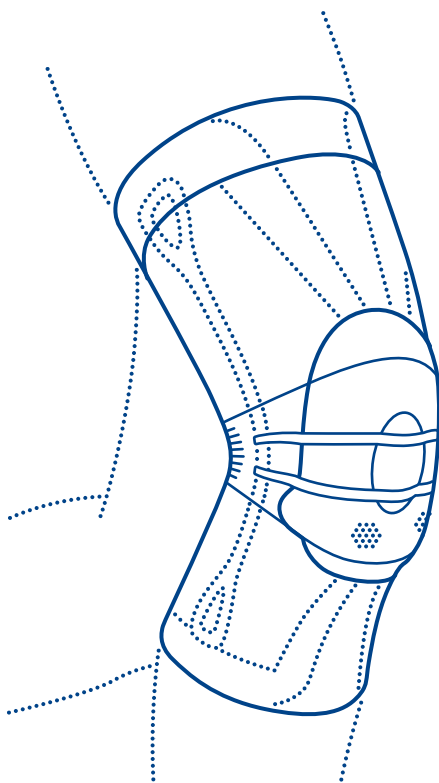
INSTRUCTIONS FOR USE

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Dear patient,
Thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these instructions for use carefully. If you have any questions, please contact your physician or medical retailer.

Intended purpose

GenuTrain is a medical device. It is a support for the relief and stabilization of the knee joint.

Indications

- Tendinopathy, tendomyopathy, ligament insertion degeneration, meniscopathy
- Chronic, post-traumatic or post-operative irritation
- Recurring joint effusion
- Osteoarthritis of the knee (joint wear)
- Arthritis (joint inflammation)
- Prevention/recurrence prevention
- Feeling of instability

Risks of using this product

Please closely observe the specifications in these instructions for use and the instructions given by the professional.

The benefits of this product are felt particularly during physical activity.

- Please doff the support during longer rest phases.
- Once you have been prescribed the GenuTrain, only use it according to your indication and any additional instructions given by a professional*. Please consult the professional or your physician first if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your physician.
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a professional before first using the product. Restrictions may exist given the specific indication which the professional can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.

Contraindications

No clinically significant adverse reactions have been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:

- Skin disorders / injuries to the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up
- Venous insufficiency, particularly with varicose veins
- Impaired sensation in the leg due to neurological abnormalities
- Circulatory disorders of the leg (e.g. in cases of diabetes mellitus)
- Impaired lymphatic drainage and soft tissue swelling of unknown origin

Application instructions

Donning of the GenuTrain

- GenuTrain with silicone top band:
- First, turn the silicone top band outward.
- Grip the donning aids at the top ends of the stays on both sides and pull the support up over the knee.

- Position the Omega+ Pad (ring-shaped pad) in such a way that it encloses the kneecap without exerting pressure. Images   

Doffing of the GenuTrain

- GenuTrain with silicone top band:
- First, turn the silicone top band outward.
- Grip the doffing aids at the bottom ends of the stays on both sides and pull the support downwards. Image 
- **Make sure not to turn the support inside-out.**

Cleaning instructions

Please use mild detergent and a mesh bag. Never expose your product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees the ideal effectiveness of the product.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the GenuTrain have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the professional were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a professional*. Before first using our medical device, please seek the advice of a physician or professional* as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this professional as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the professional, please contact your physician or distributor, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Styrene-ethylene-butylene-styrene (SEBS), Thermoplastic polyurethane (TPU), Elastane (EL), Cotton (CO), Polyurethane (PUR), Polyester (PES)
Products with silicone edging or silicone top band also contain: Silicone (SI)

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Bauerfeind Middle East FZ LLC
Bauerfeind USA, Inc.
BAUERFEIND AG