AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

Bauerfeind Benelux B.V.
waarderveldweg 1
2031 BK Haarlem
THE NETHERLANDS
P +31 (0) 23 531-9427
F +31 (0) 23 532-1970
E info@bauerfeind.nl

BELGIUM

P +32 (0) 2 527-4060
F +32 (0) 2 792-5345
E info@bauerfeind.be

BOSNIA AND HERZEGOVINA

Bauerfeind d.o.o.
Meše Seltnovića 19
71000 Sarajevo
P +387 (0) 33 710-100
F +387 (0) 33 619-422
E info@bauerfeind.ba

CHINA

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 english

Dear patient,

Thank you for choosing a Bauerfeind product.

We wish you a speedy recovery.

Please read and observe these instructions for use closely and in their entirety. If there is something you do not understand or if you have any questions, do not use the product until you have consulted your physician or professional¹ again. Keep these instructions for use in case you want to look something up later.

Intended purpose

MalleoTrain S is a medical device. It is a support for stabilization of the ankle joint thanks to a functional strap system.

Indications

- Ligament insufficiency
- Supination prevention
- Slight sprains

Contraindications

For the following clinical pictures, usage of the product is only indicated upon consultation with your physician:

- Skin disorders/lesions on the part of the body to be treated, especially if inflammatory manifestations as well as raised scars with swelling, redness and overheating are present
- Impaired sensation or circulatory disorders of the leg/foot (e.g. in cases of diabetes mellitus)
- Impaired lymphatic drainage and soft tissue swelling of unknown origin

Ask your physician whether your conditions fall into these categories.

Notes before first usage

If you wish to use your Bauerfeind product for acute symptoms, injuries or healed previous conditions, you urgently need to obtain professional medical advice and observe it before using the product. Restrictions may apply to you which you must take into account and which the professional¹ will clarify.

In case of identified or suspected problems with blood pressure and intense physical exercise, prolonged use of the product can have additional effects on your blood pressure.

After you have been prescribed/you have purchased the product, only use it according to the indication and under observance of any additional instructions given by your physician/professional¹.

If you wish to use the product in combination with medication, other medical devices or other treatments, consult your physician/professional¹ beforehand.

If any additional conditions or injuries occur during the period of application of the product, also consult your physician immediately.

Risks of application

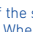
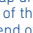
- Do not arbitrarily modify the product as this may cause it to not help as expected and/or worsen your health.
- The product must not be applied too firmly as this may cause local pressure symptoms or, in rare cases, constrict underlying blood vessels or nerves. If this happens, loosen the strap on the product.
- If you notice any changes or an increase in symptoms related to the application of the product (such as tissue congestion, skin irritation, swelling, etc.), pause the application and consult your physician/professional¹.
- If you completely doff the product, treatment is insufficient and additional damage to the part of the body affected may occur during additional strain. Be especially careful when this is the case.
- Additionally, restrictions due to the indication may apply which the professional¹ will explain to you. In particular, driving vehicles, other means of transportation or machines may only be possible under restrictions. The same applies to the use of ladders, tools and sports equipment. When in doubt, we recommend to suspend these activities.
- Persons requiring supervision due to their age or mental/physical impairment must be supervised during every application of the product.
- Avoid contact of the product with fatty and acidic agents, ointments, lotions, bases, sand or salt water.
- If hypersensitivity has been observed with you, consult your physician before the first application of the product. If hypersensitivity/allergic reaction occurs in relation with the application of the product, pause the application and consult a physician immediately.
- Furthermore, proceed with extreme caution, as your ability to stand and walk is also heavily impacted.

- Check the specific ground characteristics and ensure the ground is not slippery whenever walking, standing or otherwise stressing the affected part of the body. If the ground is smooth, there is an acute risk of slipping and thus injury!
- Never expose your product to direct sunlight, heat or cold for longer periods of time.

Application instructions

- Observe the instructions for donning and doffing of the product as well as any additional instructions for use given by your physician/professional¹ under any circumstances.
- Since the product unfolds its effects especially during physical activity, doff it during longer rest phases.
- Check the product for integrity before every use. Only intact products may be used.
- Make sure to have your physician/professional¹ show you how to don/doff the product and explain to you how it works until no doubts remain. If the affected part of the body changes in diameter, the product may have to be readjusted/newly prescribed.

Donning of the product

- Fold the top part of the support outward until you can reach the heel. Pull the support over the foot until it reaches the heel.
- Then unfold the turned-over part of the support and pull it up until the heel section is positioned correctly.
- The start of the strap is located on the inside of the foot.  When winding the strap, make sure that it sits without folds. To ensure the optimum fit of the strap, your lower leg must be perpendicular (at a right angle) to the foot during all the following steps.
- Guide the strap under the sole of your foot toward the outside of the foot.  By pulling the strap, you can generate the stabilization that's comfortable for you.
- Place the strap diagonally across the instep and back around the ankle mortise  and then guide the strap diagonally forward again to the inside of the foot.  
- Pull the strap under the sole of your foot toward the outside of the foot and over the instep  and fasten the end of the strap to the section of the strap underneath it. 

Doffing of the product

- Loosen the strap and attach the end of the strap onto another part of the strap. Take hold of the upper edge of the support and turn it down. 

Cleaning instructions

Close all the Velcro fastenings. Please use mild detergent and a mesh bag. Observe the supplementary instructions on the sewn-in label/sticker on your product. Wash the product at 30 °C/86 °F. Regular care ensures the ideal effectiveness of the product.

Area of use

Ankle joint

Notes on reuse

The product is intended for the individual treatment of one patient only.

Warranty

The legal provisions of the country in which the product was purchased apply. If you suspect a warranty claim, please first contact the person from whom you purchased the product directly. The product must be cleaned before making a warranty claim. If the instructions for handling and caring for the product have not been observed, the warranty may also be affected or excluded. The warranty does not cover:

- Off-label use
- Failure to observe the instructions of the physician/professional¹
- Arbitrary product modifications

Disclaimer

In general, do not self-diagnose, self-treat or self-medicate unless you are a professional¹. Before first using our medical device, actively consult a physician or professional¹ since this is the only way to assess the individual effect of the

product on your body and determine any possible risks of application/side effects. Please observe the advice of this professional¹ as well as all notes and instructions in this document or its online version (including texts, images, graphics, etc.), including any excerpts.

Obligation to report

Due to regional legal regulations, you are obliged to report any serious incidents involving the use of this medical device to both the manufacturer and the competent authority for medical devices (in Germany: Bundesinstitut für Arzneimittel und Medizinprodukte – Federal Institute for Drugs and Medical Devices).

Disposal

Please dispose of the product in accordance with local regulations.

Material content

Polyamide (PA), Elastane (EL), Polyurethane (PUR), Polyester (PES)

 – Medical Device

 – DataMatrix identifier as UDI

Date of information: 2026-05

¹ A professional is any person who is authorized to adjust supports and orthoses and instruct others in their use in accordance with the applicable state and institutional regulations and other guidelines. As the manufacturer, we strongly recommend that this person disposes of orthopedic technical training or comparable acquired skills, as well as orthopedic technical infrastructure.

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