**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0) 800 4430-130  
F +43 (0) 800 4430-131  
E [info@bauerfeind.at](mailto:info@bauerfeind.at)

**BENELUX**

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 english

## Dear patient,

Thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your physician or medical retailer.

## Intended purpose

MyoTrain is a medical device. It is a support with adjustable compression for the treatment of thigh muscle injuries.

## Indications

- Muscle hardening
- Muscle strains
- Muscle bruising (thigh knock)
- Muscle fiber tears
- Tears in bundles of muscle fiber, for example the rear thigh muscles (hamstrings)
- Injuries to the fascia lata
- Post-traumatic, post-operative irritation
- Prevention / recurrence prevention

## Risks of using this product

- This product is intended for the immediate treatment and prevention of subsequent injuries of the patient.
- The use and duration of use must be discussed with the physician who prescribed the MyoTrain. They will also inform you whether the MyoTrain can be used simultaneously with other products, and if so, which ones.
- Please note all the information you received with the MyoTrain as well as the instructions for use.
- The product may be damaged if it is not used correctly or is altered arbitrarily and, the warranty will not be valid.
- The effects of this support are felt primarily during physical activity. Doff the support during extended rest phases – e.g. sleeping and long periods of sitting down.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, rarely, constrict the underlying blood vessels or nerves. In such cases, please loosen the straps or the fastening slightly and have the correct fit and size of the support checked if necessary.
- If you experience any unusual changes connected to your use of the MyoTrain – e.g. an increase in discomfort – stop using it and contact your physician without delay.
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a medical professional before first using the product. Restrictions may exist given the specific indication which the specialist\* retailer can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.

 Contains natural rubber latex (elastodiene), which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

## Contraindications

- Side effects involving the body as a whole have not been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:
- Skin disorders / injuries to the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up
  - Venous insufficiency, particularly with varicose veins
  - Impaired sensation in the leg due to neurological abnormalities
  - Impaired circulation in the leg (e.g. diabetes mellitus)
  - Impaired lymphatic drainage and soft tissue swelling of unknown origin

## Application instructions

### Donning of the product

- 1 Don the MyoTrain standing up, with a relaxed leg, and a slight bend in your knee.

Don the support, making sure that the rear Velcro straps (outer) are positioned in the middle of the injured muscle area. The entire injured area must be covered by the support.

- 2 Now roll the upper edge of the support distally (downward) approx. two thirds of its length.

- 3 Place the large pad (functional pad) on the affected area of the thigh. The smaller pad (acute pad) can be inserted if you would like to increase the pressure.

Next, roll the support back up again, making sure that there are as few creases as possible.

- 4 Attach the Velcro straps to the Velcro strip on the back. The straps must be guided from bottom of the rear to the top of the front. The short straps are at the bottom and the long ones at the top.

- 5 The tensioning system (eyelets) is to be attached to the Velcro straps at the front. It is important to ensure that the straps are guided smoothly through the eyelets. 6 Starting with the lower pair of straps, tighten and fasten the inner and outer straps at the same time. The pressure of the support and the straps should be gently adjusted when they knee is bent so that the patient feels the regulated compression but this is not painful.

Once the basic adjustments have been made as described above, the support can be easily donned and doffed by opening and closing the straps. Ensure that the basic positioning of the straps is not altered.

### Use of the pads

The following items demonstrate the different applications of the insertable pads over the course of treatment:

1. Immediately after the injury:  
Place the small acute pad directly to the injury. Apply the larger functional pad on top. Both pads can be cooled in advance. Within about 24 hours, the increased compression exerted by the acute pad is no longer needed.
2. Subsequently:  
You can continue to wear the larger functional pad, particularly when resuming training. 7 You can attach the side wings of the functional pad to both sides as required.
3. In the long term:  
The MyoTrain can be worn without pads to prevent strain during sport.

## Cleaning instructions

**Note:** Never expose the support to direct heat (e.g. heaters, sunlight, leaving in your car, etc.) This may cause damage to the material, which can impair the effectiveness of the support

Please refer to the care instructions on the sewn-in label. This can be found on the upper edge of the support.

We recommend hand-washing the MyoTrain separately at 30 °C with a liquid detergent (not a heavy-duty detergent). Please note the following:

1. Remove the strap components from the knitted fabric and wash the support on its own.
2. The strap components and pads can also be washed on their own by hand as required.
3. MyoTrain is not suitable for tumble drying.

To make a complaint, please contact the distributor that provided you with the product.

**Note:** Only clean goods are handled.

## Notes on reuse

This product is intended to be fitted to one patient only.

## Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. The warranty may be limited or excluded if the instructions on how to handle and care for the MyoTrain have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

## Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a specialist\*. Before first using our medical device, please seek the advice of a physician or specialist\* as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the specialist, please contact your physician or distributor, or contact us directly.

## Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

## Disposal

Once you have finished using the product, please dispose of it according to local specifications.

## Material content

Silicone (SI), Polyamide (PA), Iron with Polyamide (Fe with PA), Elastodiene (ED), Polyurethane (PUR), Viskose (CV), Polyester (PES)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Date of information: 2025-02

\* A specialist is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

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**© 2025 Medtronic. Kõik õigused on kaitstud.**